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Stand der Information 2021-01

Ⓞ artConnect PLUS ist ein Acrylic Bonder (artVeneer® - artDentine PLUS) especially for highly cross-linked acrylic veneers and PMMA-based acrylic teeth for bonding to artDentine PLUS. Also recommended for preparation of mould correction or supplementation of artVeneer®.

Indication

- Total or partial tooth loss
- Loss of hard tooth substance

Replacement of lost hard tooth substance by:

- Tooth-shaded fixture and for backing artVeneer®
- Veneering telescopic, attachment, bar and implant constructions with artVeneer®
- Tooth-shaded backing of trimmed acrylic teeth
- Fixing acrylic teeth to the model cast with a back protection plate or pontic
- Individualising acrylic teeth, artVeneer® and CAD/CAM restorations produced from M-PM® Disc and the artBloc® Temp
- Repairs

Contraindication

- Hypersensitivity to one of the components.

Components of the artConcept® PLUS system

- artPrime PLUS - Metal bonding agent
- artPreOpaque PLUS - Priming opaqueur
- artOpaque PLUS - Opaqueur for covering framework
- artDentine PLUS/artEnamel PLUS Polymer - Tooth-shade PMMA Acrylic
- artDentine PLUS Liquid NT (normal time) / artDentine PLUS Liquid LT (long time)
- Monomer component for artDentine PLUS / artEnamel PLUS

- artConnect PLUS - Acrylic bondor** artVeneer® - Tooth-shade PMMA anterior and posterior veneer

Composition

MMA (Methyl methacrylate), THFMA (Tetrahydrofurfuryl methacrylate)

Processing

Fixture of the veneers

Wetten the surface to be joined with artConnect Plus and apply sufficient that the connecting surface remains moist for 5 min to work in. If the connecting surface with artConnect Plus dries out during this time, moisten again with artConnect Plus. The exposure time must not be less than 5 min; the fluid artDentine PLUS resin has to be applied to the conditioned surface after 10 min at the latest.

Storage

Store in a dry and cool area at 10 -25 °C / 50 -77 °F without direct exposure to light.

Disposal

The fully polymerized components of artConcept® PLUS are disposed of as residual waste. Disposal of non-polymerized product residues according to legal regulations. Do not allow entry into the sewage system or waterways.

Notes

- Close all containers tightly after use.
- Do not use the material after the specified expiration date.
- The residual monomer content after polymerisation of artDentine PLUS/ artEnamel PLUS is < 1,8 %.
- For more information on processing acrylics and reducing residual monomer content, please visit www.merz-dental.de/service

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The product properties are based on compliance with and observation of these instructions for use.

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