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en Nesting recommendation for M-ZR multicolor ST and M-ZR multilayer HT+

Our M-ZR multicolor ST and M-ZR multilayer HT + zirconia blanks guarantee shades according to VITA® and a smooth shade gradient. The chroma is reduced in the incisal area and significantly lightened in order to achieve not only a shade gradient but also a translucency gradient. The vibrant chroma and an increase in translucency through a defined light refraction in the incisal third use the full potential of the cubic zirconia and thus fulfill the highest demands of CAD materials. A natural appearance for highly aesthetic full contour restorations - quick and easy.

A sophisticated production technique ensures the reproducibility of layers in the blank.

By transferring our layering scheme to the nesting software (CAM software or exocad® „In-CAD Nesting Module“) you increase planning reliability. The visually supported work process on the screen gives you full control over the brightness value and chroma of your crowns and bridges in the anterior and posterior area.

Layer heights, pre-sintered for your nesting software	Blank height (mm)	Layer 1+2: Incisal (mm / %)	Layer 3: Intermediate (mm / %)	Layer 4: Intermediate (mm / %)	Layer 5: Body / Dentine (mm / %)
	14 mm	3.5 / 24.9	2.1 / 15	2.1 / 15	6.3 / 45.1
	18 mm	3.5 / 19.4	2.1 / 11.7	2.1 / 11.7	10.3 / 57.2
	22 mm	3.5 / 15.9	2.1 / 9.6	2.1 / 9.6	14.3 / 64.9

Optimum CAM-Nesting: Indication: Crown, tooth 46, shade A3 – Blank: M-ZR multicolor ST, Height 14 mm, shade A3

	Blank position 1.0 mm	Restoration shade gradient	Blank position 2.5 mm	Restoration shade gradient	Blank position 3.3 mm	Restoration shade gradient
<p>You will achieve the best shade match by individually positioning the restoration in the blank (match).</p> <p>Depending on the height of the restoration, the positioning of the incisal, intermediate and dentine layer can be individually adjusted to achieve the best possible shade gradient.</p>		Intense brightening also in the body of the crown. Shade is closer to A2.		Brightening in the cusps. Shade gradient perfectly matched to the ratio of crown height.		No brightening in the cusps. Chroma is darkened.
	too bright		match		too dark	

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