Baltic Denture System

BDLoad®

- | | |
|--------------------------------------|------------------------------------|
| (de) Gebrauchsanweisung | (fi) Käyttöohje |
| (en) Instructions for use | (it) Naudojimo instrukcija |
| (usa) Instructions for use | (lv) Lietošanas informācija |
| (fr) Notice d'utilisation | (et) Kasutusjuhend |
| (it) Istruzioni per l'uso | (hu) Használati utasítás |
| (es) Instrucciones de uso | (cs) Navos k použití |
| (pt) Instruções de utilização | (sl) Navodila za uporabo |
| (pl) Instrukcja użycia | (sk) Návod na používanie |
| (nl) Gebruiksaanwijzing | (bg) Инструкции за употреба |
| (sv) Bruksanvisning | (el) Οδηγίες χρήσης |
| (da) Brugsanvisning | (tr) Kullanım kılavuzu |
| (no) Bruksanvisning | |

MADE IN GERMANY

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Stand der Information 2021-08

Milling blank with integrated occlusion concept based on PMMA

en Instructions for use, please read carefully!

Intended purpose

BDLoad®* serves to fabricate upper and / or lower jaw full dentures by milling for temporary and final restorations.

The BDLoad® TS, consisting of tooth-coloured PMMA, is used exclusively for the fabrication of temporary restorations fabricated with the milling technique, such as for example:

- maxillary and / or mandibular try-ins (with wax veneer if necessary)
- milled individual impression trays
- maxillary and / or mandibular interim prostheses (max. 3 months)

Classification according to MDD 93/42/EEC and MDR (EU) 2017/745

- Class IIa medical device for long-term contact with intact mucous membranes

Intended users

- Dental professionals

Patient target groups

- Adolescents and adults

Indication

- Edentulism of one or both jaws with a eugnathic jaw situation

Contraindication

- Partially edentulous jaw situation
- Dysgnathia
- Intolerance to one of the components

Composition BDLoad® / BDLoad® ECO / BDLoad® TS / BDLoad® XP

- Cross-linked polymethylmethacrylate (PMMA) copolymers
- Pigments (primarily iron and titanium oxide)
- Dibenzoylperoxide may be contained as residual peroxide < 1 %
- Methylmethacrylate (MMA)

* In the following text, BDLoad® always refers to the entire product group consisting of the variants BDLoad® ECO, BDLoad® High Impact (HI), BDLoad® High Impact ECO (HI ECO), BDLoad® Tooth Shade (TS), BDLoad® XP, BDLoad®, unless the corresponding variant is explicitly named.

Composition ^{BD}Load[®] High Impact / ^{BD}Load[®] High Impact ECO

Base:

- impact-modified polymethylmethacrylate (PMMA) copolymers
- Pigments (primarily iron and titanium oxide)
- Dibenzoylperoxide may be contained as residual peroxide < 1 %
- Methylmethacrylate (MMA)

Teeth:

- Cross-linked polymethylmethacrylate (PMMA) copolymers

Use is only admissible by trained technical personnel.

Digital design

- The ^{BD}Load[®] XP allows an individual anterior tooth to be set up in the software.
- The denture base is designed and generated using the ^{BD}Creator[®] PLUS software.
- A minimum wall thickness of 2 mm is specified by the software.

Machine processing

Both wet and dry milling processes are possible with the suitable clamping system.

Compatibility should be clarified with the respective machine manufacturer prior to processing.

- ^{BD}Load[®] can only be used and milled in a clamping device approved by the respective machine manufacturer for this geometry.
- Machine processing of the ^{BD}Load[®] requires special CAM software and milling cutters suitable for machining PMMA.
- Tooth shape, ^{BD}Load[®] size and shade are stated on the ^{BD}Load[®] primary marking and must match the specifications in the software.
- The patient-specific antagonist situation is taken into account when fabricating a unimaxillary restoration. Here the occlusal surfaces are machined in a subtractive manner.

Individual set-up of the anterior teeth (^{BD}Load[®] XP)

The anterior teeth can be set up individually by

- Selecting integrated, predefined tooth set-ups in ^{BD}Creator[®] PLUS.
- Individual, digital set-up of the anterior teeth in the ^{BD}Creator[®] PLUS.

To manually change the position of the anterior teeth, e.g. during a try-in, the PMMA wall must be reduced in the area of the milled tooth compartments using suitable cross-toothed carbide burs. The adjusted set-up is then fixed with wax in the desired position.

Milling of tooth compartments is required!

Irrespective of the workflow, the anterior teeth are bonded to the ^{BD}Load[®] XP using the matrix technique with cold polymerisate, e.g. Weropress[®] in the ^{BD}Load[®] pink shade.

Trimming following the milling process

- Cross-cut carbide burs suitable for resin should be used to separate and rework denture bases from the ^{BD}Load[®].
- To prevent inaccurate fitting after production, avoid excessive heat development during trimming and polishing.
- The wall thickness for tapered thin surfaces following finishing should not be below a minimum of 1.5 mm if possible.

Restoration

Denture bases produced from ^{BD}Load[®] can be restored, supplemented or lined at any time with MMA-based cold-curing polymers, such as Weropress[®] or Combipress N in the matching shade. Prior conditioning of the surface is necessary, preferably with artConnect PLUS or a similar acrylate-based bonding agent in accordance with the specifications of the manufacturer.

Storage

- ^{BD}Load[®] has to be stored in the packaging to protect the integrated row of teeth.
- Protect against direct sunlight.

Notes

- Record the LOT number with every process that requires identification of the material; the detachable label must be kept for the purposes of traceability.
- Do not use ^{BD}Load[®] after the specified expiration date.
- Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery.
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

Risk phrases

- Processing ^{BD}Load[®] generates dust that can irritate the eyes, skin and airways. Therefore always ensure that the extraction on your milling machine and for finishing and polishing with a handpiece or polishing unit is working correctly.
- The use of personal protective equipment in accordance with the Material Safety Data Sheet must be observed.
- Improper use of ^{BD}Load[®] with the aim of extending the above indication areas is inadmissible.

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from www.merz-dental.de

System components

- The list of approved system components is available at www.baltic-denture-system.de in the **Service** section.
- The use of non-authorized system components in conjunction with the Baltic Denture System is inadmissible.

Disposal

Residues from processing the ^{BD}Load[®] have to be disposed of as residual waste.

Printed copies of these instructions for use can be requested free of charge using the contact details provided.

The users themselves are responsible for the use of the product. Merz Dental assumes no liability and / or guarantee for the use of components that are not part of the system and / or for components that have not been tested and for incorrect results, as the manufacturer has no influence on processing. Any claims for damages that may still arise relate exclusively to the value of our products.

The product properties are based on compliance with the instructions for use.

Date of information 2021-08

Milling blank with integrated occlusion concept based on PMMA

usa Instructions for use, please read carefully!

Caution:

Federal (USA) law restricts this device by or on the order of a licensed dental practitioner.

Intended purpose

BDLoad®* serves to fabricate upper and / or lower jaw full dentures by milling for temporary and final restorations.

The BDLoad® TS, consisting of tooth-coloured PMMA, is used exclusively for the fabrication of temporary restorations fabricated with the milling technique, such as for example:

- maxillary and / or mandibular try-ins (with wax veneer if necessary)
- milled individual impression trays
- maxillary and / or mandibular interim prostheses (max. 3 months)

Classification according to MDD 93/42/EEC and MDR (EU) 2017/745

- Class IIa medical device for long-term contact with intact mucous membranes

Intended users

- Dental professionals

Patient target groups

- Adolescents and adults

Indication

- Edentulism of one or both jaws with a eugnathic jaw situation

Contraindication

- Partially edentulous jaw situation
- Dysgnathia
- Intolerance to one of the components

Improper use of BDLoad® with the aim of extending the above indication areas is inadmissible. Improper use of BDLoad® can impair the health of the patients treated.

* In the following text, BDLoad® always refers to the entire product group consisting of the variants BDLoad® ECO, BDLoad® High Impact (HI), BDLoad® Tooth Shade (TS), BDLoad® High Impact ECO (HI ECO), BDLoad®, unless the corresponding variant is explicitly named.

Composition ^{BD}Load® / ^{BD}Load® ECO / ^{BD}Load® TS

- Cross-linked polymethylmethacrylate (PMMA) copolymers
- Pigments (primarily iron and titanium oxide)
- Dibenzoylperoxide may be contained as residual peroxide < 1 %
- Methylmethacrylate (MMA)

Composition ^{BD}Load® High Impact / ^{BD}Load® High Impact ECO

Base:

- impact-modified polymethylmethacrylate (PMMA) copolymers
- Pigments (primarily iron and titanium oxide)
- Dibenzoylperoxide may be contained as residual peroxide < 1 %
- Methylmethacrylate (MMA)

Teeth:

- Cross-linked polymethylmethacrylate (PMMA) copolymers

Use is only admissible by trained technical personnel.

Digital design

- The denture base is designed and generated using the ^{BD}Creator® PLUS software.
- A minimum wall thickness of 2 mm is specified by the software.

Machine processing

Both wet and dry milling processes are possible with the suitable clamping system.

Compatibility should be clarified with the respective machine manufacturer prior to processing.

- ^{BD}Load® can only be used and milled in a clamping device approved by the respective machine manufacturer for this geometry.
- Machine processing of the ^{BD}Load® requires special CAM software and milling cutters suitable for machining PMMA.
- Tooth shape, ^{BD}Load® size and shade are stated on the ^{BD}Load® primary marking and must match the specifications in the software.
- The patient-specific antagonist situation is taken into account when fabricating a unimaxillary restoration. Here the occlusal surfaces are machined in a subtractive manner.

Trimming following the milling process

- Cross-cut carbide burs suitable for resin should be used to separate and rework denture bases from the ^{BD}Load[®].
- To prevent inaccurate fitting after production, avoid excessive heat development during trimming and polishing.
- The wall thickness for tapered thin surfaces following finishing should not be below a minimum of 1.5 mm if possible.

Restoration

Denture bases produced from ^{BD}Load[®] can be restored, supplemented or lined at any time with MMA-based cold-curing polymers, such as Weropress[®] or Combipress N in the matching shade. Prior conditioning of the surface is necessary, preferably with artConnect PLUS or a similar acrylate-based bonding agent in accordance with the specifications of the manufacturer.

Storage

- ^{BD}Load[®] has to be stored in the packaging to protect the integrated row of teeth.
- Protect against direct sunlight.

Notes

- Record the LOT number with every process that requires identification of the material; the detachable label must be kept for the purposes of traceability.
- Do not use ^{BD}Load[®] after the specified expiration date.
- Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery.
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

Risk phrases

- Processing ^{BD}Load[®] generates dust that can irritate the eyes, skin and airways. Therefore always ensure that the extraction on your milling machine and for finishing and polishing with a handpiece or polishing unit is working correctly.
- The use of personal protective equipment in accordance with the Material Safety Data Sheet must be observed.
- Improper use of ^{BD}Load[®] with the aim of extending the above indication areas is inadmissible.

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from www.merz-dental.de

System components

- The list of approved system components is available at **www.baltic-denture-system.de** in the **Service** section.
- The use of non-authorised system components in conjunction with the Baltic Denture System is inadmissible.

Disposal

Residues from processing the ^{BD}Load[®] have to be disposed of as residual waste.

Printed copies of these instructions for use can be requested free of charge using the contact details provided.

The users themselves are responsible for the use of the product. Merz Dental assumes no liability and / or guarantee for the use of components that are not part of the system and / or for components that have not been tested and for incorrect results, as the manufacturer has no influence on processing. Any claims for damages that may still arise relate exclusively to the value of our products.

The product properties are based on compliance with the instructions for use.

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