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Stand der Information 2022-04

**(en) Instructions for use, please read carefully!**

### Regulatory information

<sup>mp</sup>Illusion Color and <sup>mp</sup>Illusion Paste meet all requirements of applicable directives and regulations on medical devices, and the manufacturing is certified acc. ISO 13485 and annex II of MDD 93/42/EWG, VO (EU) 2017/745 and further international requirements.

Medical device classification acc. annex IX, rule 8 of MDR 93/42: IIa

Medical device classification acc. annex VIII, rule 8 of MDR 2017/745: IIa

MDR Code acc. MDCG 2019-14: MDT 2003, MDN 1103

Classification acc. DIN EN ISO 6872: type 1, class 1

### Indication

Reconstruction of natural teeth (crowns and bridges) made from following framework materials:

1. monolithic or layered zirconium dioxide (Y-TZP) crowns and bridges with thermal expansion of  $10.6 \times 10^{-6} \times K^{-1}$  (25-500 °C).
2. Lithium disilicate (monolithic / veneered) with a thermal expansion of approx.  $10.0 \times 10^{-6} \times K^{-1}$  (25-500 °C).
3. Frameworks made of pressed ceramics with a CTE value of approx.  $9.5 \times 10^{-6} \times K^{-1}$
4. Metal ceramic with a CTE value of approx.  $13.0 \pm 0.5$  (25-500 °C) [ $10^{-6} K^{-1}$ ].

### Contraindications:

1. Combinations with materials outside of the product system described.
2. Veneering of framework materials that are not indicated.
3. Sharp edges and corners on the framework.
4. Dental ceramic and all ceramic restorations made of glass ceramics are not recommended for patients with bruxism or parafunction.

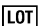
### Chemical composition <sup>mp</sup>Illusion Paste:

SiO<sub>2</sub>, Al<sub>2</sub>O<sub>3</sub>, K<sub>2</sub>O, Na<sub>2</sub>O, Li<sub>2</sub>O, SrO, B<sub>2</sub>O<sub>3</sub>, CeO<sub>2</sub>, ZnO + pigments

### Chemical composition <sup>mp</sup>Illusion Color:

SiO<sub>2</sub>, Al<sub>2</sub>O<sub>3</sub>, K<sub>2</sub>O, Na<sub>2</sub>O, Li<sub>2</sub>O, CaO, SrO, B<sub>2</sub>O<sub>3</sub>, ZnO, F + pigments

### Explanation of symbols on the product labels

 Manufacturer	 Batch code
 Date of manufacture YYYY-MM	 Reference number
 Medical device	 Caution, consult instruction for use

## Physical properties

### **Illusion Paste**

Physico-chemical properties according to DIN EN ISO 6872

Property	Specification	Measurement values
Coefficient of thermal expansion (25 - 475 °C) [ $\cdot 10^{-6} \cdot K^{-1} \pm 0.5$ ]	2 x: 9.5 4 x: 9.5	2 x: 9.5 4 x: 9.5
Glass transition temperature T <sub>g</sub> [°C ± 20]	2 x: 495 4 x: 495	2 x: 495 4 x: 495
Flexural strength [MPa]	≥ 50	130 - 150
Solubility [µg/cm <sup>2</sup> ]	< 100	19 - 35

### **Illusion Color**

Physico-chemical properties according to DIN EN ISO 6872

Property	Specification	Measurement values
Coefficient of thermal expansion (25 - 450 °C) [ $\cdot 10^{-6} \cdot K^{-1} \pm 0.5$ ]	2 x: 10.0 4 x: 10.0	2 x: 9.8* 4 x: 9.5*
Glass transition temperature T <sub>g</sub> [°C ± 20]	2 x: 460 4 x: 460	2 x: 455* 4 x: 455*
Flexural strength [MPa]	≥ 50	> 130*
Solubility [µg/cm <sup>2</sup> ]	< 100	complies with*

\*) data for base material

### **Important processing instructions**

- Only to be used by trained personnel.
- For use in clean working environments only! Contamination of the pastes, the mixing tray, the preheating furnace or any additional materials as waxes or liquids especially with CoCr-alloy residues may cause discoloration of restorations.

### **Recommended storage conditions**

- 12 - 38° C and normal air humidity 40 - 60%.
- Storage in tightly closed original containers.
- Protect from direct sunlight.

### **Application instructions**

- Before each application, mix the <sup>m</sup>Illusion Color and <sup>m</sup>Illusion Paste compounds well with a metal-free spatula.
- Always use a clean and dry spoon, spatula or brush.
- Moisten brush with <sup>m</sup>Illusion Paste liquid before application.
- Only use <sup>m</sup>Illusion Paste Liquid to adjust the consistency of both the <sup>m</sup>Illusion Color and <sup>m</sup>Illusion Paste compounds.
- **Note:** The pastes must not come into contact with water.
- Do not fill removed and used pastes back into the container. Use clean and dry instruments for removal.

### **Warnings**

- When working on ceramic restorations safety glasses should be used. Remove dust and fragments by suction or wear a proper dust protection mask.
- Be careful of high firing and pressing temperatures. Danger of getting burnt! Use oven pincers and gloves!

### **Framework preparation**

- Prepare the frameworks according to the manufacturer's instructions.
- Before the first application of <sup>m</sup>Illusion Color or <sup>m</sup>Illusion Paste, carefully sandblast the surface with 110 µm aluminum oxide and 2 bar pressure.
- Steam clean the objects before each application.

### **Colouring**

- Before colouring, apply a thin layer of <sup>m</sup>Illusion Liquid to the surface.

### **Structuring**

- Before applying <sup>m</sup>Illusion Paste for the first time, apply a thin layer of <sup>m</sup>Illusion Paste Liquid to the surface.

### **Layering/staining**

Apply <sup>m</sup>Illusion product components to the crown or bridge as desired.

### **Applying the glaze/polishing**

After finishing the restoration, apply a thin layer of <sup>m</sup>Illusion glaze fluor paste to the entire crown and fire. The gloss level of the restoration can be adjusted after the glaze firing using a polishing wheel and polishing paste.

## FIRING TABLE

Due to the different ceramic furnaces available on the market, the firing conditions may differ. This must be taken into account and is under the responsibility of the customer!

The specified firing temperatures are **only reference values!**

**Note:** The indicated firing temperatures are only APPROXIMATE VALUES tested in a Zubler Vario 300 ceramic furnace. Adjustments to the firing program may be necessary for other furnace types.

Firing parameters	Start-temperature	Pre-Dry time	Closing time	Heating rate	Vakuum start	Final temperature	Holding time
	(°C)	(min)	(min)	(°C/min)	(°C)	(°C)	(min)
Monolithic zirconium dioxide (Y-TZP)	400	3	4	45	670	770	1
Metal ceramic	400	3	4	45	670	770	1
Pressed ceramic	400	3	4	45	670	770	1
Lithium disilicate (monolithic)	400	3	4	45	670	720	2
Lithium disilicate (veneered)	400	3	4	45	670	720	2
Zirconium dioxide (Y-TZP), veneered	400	3	4	45	670	720	2

Please note: please use retarded furnace opening time (2 min.) for voluminous reconstructions!

**Note:** Reduce final temperature when firing multiple times for about 10 – 20° C, according to the surface conditions (gloss)!

---

For detailed information, please refer the applicable Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de)

---

**The product properties are based on compliance with and observance of these instructions for use.**

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**Merz Dental GmbH**