artPrime PLUS

Metal Primer

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(en) artPrime PLUS is a universal metal-adhesive primer which promotes adhesion between dental alloys and resin material.
Indication

- Total or partial tooth loss
- Loss of hard tooth substance

Intended use

Replacement of lost hard tooth substance by:
• Tooth-shaded fixture and for backing artVeneer®
• Veneering telescopic, attachment, bar and implant constructions with artVeneer®
• Tooth-shaded backing of trimmed acrylic teeth
• Fixing acrylic teeth to the model cast with a back protection plate or pontic
• Individualising acrylic teeth, artVeneer® and CAD/CAM restorations produced from M-PM® Disc and the artBloc® Temp
• Repairs

Contraindication

• Hypersensitivity to one of the components Components of the artConcept® PLUS system

• artPrime PLUS - Metal bonding agent

• artPreOpaque PLUS - Priming opaquer

• artOpaque PLUS - Opaquer for covering framework

• artDentine PLUS/artEnamel PLUS

• artDentine PLUS Liquid NT (normal time) / artDentine PLUS Liquid LT (long time) - Monomer component for artDentine PLUS / artEnamel PLUS

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Composition

Acetone, phosphoric acid monomer, thioctic acid

Processing

1. Bonding surface treatment

Sandblast the metal bonding surface with 50 - 100 µm alumina particles. Rinse with water and dry.

2. Application

Place a sufficient quantity of artPrime PLUS in a mixing bowl; using a disposable brush, apply a layer over the entire adhesive surface and leave for 10 seconds to take effect.

3. Perform bonding or build-up procedures using resin material according to its instruction manual.

Storage

Keep away from flame, ignition source or direct sunlight and store at room temperature(1 - 30 °C/ 34 - 86 °F).

Disposal

The fully polymerized components of artConcept® PLUS are disposed of as residual waste. Disposal of non-polymerized product residues according to legal regulations. Do not allow entry into the sewage system or waterways.

Notes

• Close all containers tightly after use.

• Do not use the material after the specified expiration date.

• To reduce the residual monomer content, the finished (sanded) but not polished polymerized workplace should be stored in water at room temperature for about 48 hours before delivery.

• The residual monomer content after polymerisation of artDentine PLUS/artEnamel PLUS is < 1.8 %.

Danger and safety instructions

(Contens <125ml)

H336 May cause drowsiness or dizziness

P235 Keep cool.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from www.merz-dental.de.

The product properties are based on compliance with and observation of these instructions for use.

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