Combitray[®] LC

Löffelmaterial
Basisplatten

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- | | |
|-------------------------------|-----------------------------|
| (en) Instructions for use | (lt) Naudojimo instrukcija |
| (fr) Notice d'utilisation | (lv) Lietošanas informācija |
| (it) Istruzioni per l'uso | (et) Kasutusjuhend |
| (es) Instrucciones de uso | (hu) Használati utasítás |
| (pt) Instruções de utilização | (cs) Návod k použití |
| (pl) Instrukcja użycia | (sl) Navodila za uporabo |
| (nl) Gebruiksaanwijzing | (sk) Návod na používanie |
| (sv) Bruksanvisning | (bg) Инструкции за употреба |
| (da) Brugsanvisningen | (el) Οδηγίες χρήσης |
| (no) Bruksanvisningen | (tr) Kullanım kılavuzu |
| (fi) Käyttöohje | |

Combitray® LC Löffelmaterial Combitray® LC Basisplatten

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Combitray® LC Tray Material

Combitray® LC Base Trays

Light curing composite resin for the fabrication of custom trays and base plates

(en) Instructions for use, please read carefully!

Intended use

Medical device for dental use for fabricating individual impression trays, base plates for bite templates, and plates for intraoral support pin registrations.

User profile

Combitray® LC is a product for professional use by dental technicians or dentists. Intended for use in the dental laboratory or dental practice laboratory. No special training is required. The finished products made from the material are intended for professional use by dentists in a dental practice. No special training is required.

Patient group

The use of Combitray® LC is not limited to any particular patient group. Contraindications must be observed.

Indication

Combitray® LC Tray Material

- Precision impressions for accurately fitting restorations with the highest marginal accuracy, such as
- functional impressions
- Situation impressions for unfavorable dental arch and jaw shapes and/or deep-set bands

Combitray® LC Base Trays

- Bite registrations
- Support pin registrations

Contraindication

- Hypersensitivity to one of the components

Undesirable effects:

- Combitray® LC contains methacrylates. They may cause an allergic skin reaction. In rare cases sensitisation is possible.

Composition

- Acrylates / methacrylates
- Glass beads
- Pyrogenic silicon dioxide
- Photochemical initiators
- Petrolatum

Areas of Application

Combitray® LC Tray Material (thickness 2.2 mm)

- Fabrication of form-stable individual impression trays
- Fabrication of base plates for bite templates
- Plates for intraoral bearing devices

Combitray® LC Base Trays (thickness 1,4 mm)

- As a hard base for wax set-ups
- As a hard base for impression templates
- Plates for intraoral bearing devices

Application

- The situation model should be prepared as required and insulated with an alginate insulation (e.g. PremEco® Line). Undercutting areas of the model should be blocked out before.
- Open the can at the handle tab, remove one Combitray® LC plate incl. protective fleece and immediately reseal the can tightly.
- The protective fleece is removed from the Combitray® LC sheet. The sheet is adapted to the insulated model and trimmed at the edges.
- The handle can be formed from the excess material. The handle is not used when making a base plate.
- The sheet is then polymerized on the model in a blue light or UVA light device for 5 minutes. The instructions for use of the light unit used must be observed!
The sheet is removed from the model and polymerized from the back for another 5 minutes.
- The individual impression tray made in this way can be processed immediately.
- The inhibition layer (adhesive layer on the top of the material) allows subsequent correction (e.g. reinforcement of the sheet). The inhibition layer can be easily removed with a moistened tissue with an alcoholic cleaning agent.

Shelflife

- 3 years from manufacture. Do not use after the expiry date stated on the label!

Storage

- Keep containers tightly closed and store protected from light. Avoid direct exposure to sunlight.
- Store at 4-25°C.

Special Notes

- Carefully close Combitray® LC after use
- Record the LOT number with every process that requires identification of the material
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

Risk phrases

H317 May cause an allergic skin reaction.

Safety notes

P261 Avoid breathing dust/fumes/gas/mist/vapours/spray

P280 Wear protective gloves/protective clothing/eye protection/face protection

P302 + P352 If on skin: Wash with plenty of water

P333 + P313 If skin irritation or rash occurs: Get medical advice/attention.

When grinding the polymerized plates, use a suitable dust extractor and follow the instructions for use of the abrasive manufacturer, especially with regard to dust protection. Use self-contained respirator, as appropriate.

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from www.merz-dental.de

Disposal

Do not allow entry into the sewage system or waterways. Do not allow entry into the soil or subsoil. Disposal according to legal regulations.

Printed copies of these instructions for use can be requested free of charge using the contact details provided.

The users themselves are responsible for the use of the product. Merz Dental assumes no liability and / or guarantee for the use of components that are not part of the system and / or for components that have not been tested and for incorrect results, as the manufacturer has no influence on processing. Any claims for damages that may still arise relate exclusively to the value of our products.

The product properties are based on compliance with and observation of these instructions for use.

Date of information 2021-06-30

Symbols

REF Article number

Protect from the sun

LOT Batch

Observe Instructions for use (IfU)

 Use by

Temperature limitation

MD Reference that the product is a medical device

Safety notes in the IfU

 Manufacturer

May cause an allergic skin reaction

Supply Unit

Tray Material 50 sheets, pink
Base Trays 50 sheets, pink

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MEGADENTA Dentalprodukte GmbH
Merz Dental GmbH