# M-PM<sup>®</sup> crystal

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- |   |   |
|---|---|
| <b>(USA)</b> <b>(en)</b> Instructions for use | <b>(lt)</b> Naudojimo instrukcija       |
| <b>(fr)</b> Notice d'utilisation              | <b>(lv)</b> Lietošanas informācija      |
| <b>(it)</b> Istruzioni per l'uso              | <b>(et)</b> Kasutusjuhend               |
| <b>(es)</b> Instrucciones de uso              | <b>(hu)</b> Használati utasítás         |
| <b>(pt)</b> Instruções de utilização          | <b>(cs)</b> Návod k použití             |
| <b>(pl)</b> Instrukcja użycia                 | <b>(si)</b> Navodila za uporabo         |
| <b>(nl)</b> Gebruiksaanwijzing                | <b>(sk)</b> Návod na používanie         |
| <b>(sv)</b> Bruksanvisning                    | <b>(bg)</b> Инструкции за употреба      |
| <b>(da)</b> Brugsanvisningen                  | <b>(el)</b> Οδηγίες χρήσης              |
| <b>(no)</b> Bruksanvisningen                  | <b>(tr)</b> Kullanım kılavuzu           |
| <b>(fi)</b> Käyttöohje                        | <b>(uk)</b> Інструкція для застосування |

# M-PM<sup>®</sup> crystal

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Stand der Information 2023-03

# M-PM<sup>®</sup> crystal

Milling blank made of thermoelastic PMMA without fillers and peroxide

**USA en** Instructions for use, please read carefully!

**Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed dental practitioner and dental technician.**

## **Intended purpose**

For the fabrication of occlusal splints, drilling templates, functional impression trays and impression trays, bite templates, bleaching trays and "lost moulds" for casting in dental prosthetics.

## **Classification according to MDD 93/42/EEC and MDR (EU) 2017/745**

Class IIa medical device for long-term contact with intact mucous membranes or temporary contact with injured mucous membranes.

## **Patient target groups**

All genders and age groups

## **Intended users**

Dental professionals

## **Indication**

- Abnormal jaw function
- Total or partial tooth loss
- Bruxism

## **Contraindication M-PM crystal**

Hypersensitivity to one of the components

## **Composition**

- Polymethylmethacrylate (PMMA)
- Methylmethacrylate (MMA) may be contained as residual monomer up to max. 1%.

## **CAM processing**

- In dry processing in the milling machine, care must be taken that milling chips are extracted or evacuated.
- M-PM crystal can generally be used with the common burs available for thermoplastics and PMMA. If you have questions on these topics, please contact the manufacturer of your milling machine, software or milling tools.

- M-PM crystal features a glass-transition temperature of 121 °C. Particularly in the case of dry milling, it should be ensured that there is no excessive heat transfer to the material, e.g. due to excess speed and/or feed rate, as otherwise the milling tool and/or the material surface could smear.

### Physical data

Property	Result	Test method
Glass transition	121 °C	ISO 11357
Flexural strength	156 ± 6 MPa	EN ISO 10477
Modulus of elasticity	3000 ± 100 MPa	EN ISO 20795-1
Vickers hardness	23.2 ± 0.3 HV 0.2	EN ISO 6501
Water uptake	< 32 µg / mm <sup>3</sup>	EN ISO 20795-1
Water solubility	< 1.6 µg / mm <sup>3</sup>	EN ISO 20795-1

### Trimming

- Cross-cut carbide burs suitable for resin should be used to rework and separate frameworks, crowns, bridges, denture bases, bite splints and drilling templates from the blank.
- To prevent inaccurate fitting after fabricating the construction, avoid excessive heat development during trimming and polishing.

### Restoration M-PM crystal

Constructions produced from M-PM crystal can be restored, supplemented or lined at any time with MMA-based cold-curing polymers, such as Weropress in the matching shade. Following roughening of the surface (sanding / blasting / grinding), further conditioning is recommended with an MMA-based bonding agent.

### Storage

No special measures required.

### Notes

- Minimum wall thickness after polishing ≥1 mm.
- Do not use the material after the specified expiration date.
- Record the LOT number with every process that requires identification of the material.
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

- To ensure traceability and assignment of the product data as well as the LOT numbers for the patient at any time, it is recommended to store the milling blank in the original packaging during two milling operations.
- When disinfecting the construction, only alcohol-free disinfectants should be used as crazing could otherwise form.
- Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery.

### **Warnings**

Processing PMMA frameworks may generate dust which can lead to mechanical irritation of the eyes and airways. Therefore, always ensure that the extraction system on your milling machine and at the workplace is working properly for individual post-processing and that you are using personal protective equipment.

---

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de).

---

### **Disposal**

Residues from processing the M-PM crystal have to be disposed of as residual waste.

**The product properties are based on compliance with and observation of these instructions for use.**

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**Merz Dental GmbH**