M-ZR

Basic Shade HC Pro Shade LC Art Shade

- Ⓞ de Gebrauchsanweisung
- Ⓞ en Instructions for use
- Ⓞ fr Notice d'utilisation
- Ⓞ it Istruzioni per l'uso
- Ⓞ es Instrucciones de uso
- Ⓞ pt Instruções de utilização
- Ⓞ nl Gebruiksaanwijzing
- Ⓞ el Οδηγίες χρήσης

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en Instructions for use, please read carefully!

Intended use

M-ZR Basic Shade HC, M-ZR Pro Shade LC and M-ZR Art Shade are liquids for coloring pre-sintered zirconia frameworks made from Merz Dental zirconia milling blanks for the manufacture of dental restorations.

Indication

Loss of hard tooth substance.

Contraindication

Do not use in case of intolerance towards the ingredients contained. Please also consider the contraindications for the respective Merz Dental zirconia used. The staining solutions are not suitable for zirconia ceramics from other manufacturers.

Composition

Aqueous hydrochloric acid solution, contains:

erbium(III) chloride

iron(III) chloride hexahydrate

Processing/Notes

This medical product may only be handled by trained personnel. Only process defect-free, dry frameworks made from Merz Dental zirconia (ZrO_2) milling blanks.

Processing/Trimming

Immersion technique :

- Select the coloring liquid according to the framework material and the color to be achieved (shade combination guide can be found in the full instructions at www.merz-dental.de)
- Remove any residual dust from the ZrO_2 framework by careful cleaning with compressed air. Zirconia frameworks processed in wet-milling machines must be dried immediately after the wet-milling process
- Work place preparation: Prepare a timer, paper cloth, plastic forceps and clean immersion container plus an applicator
- Shake liquid bottles well before use
- Fill an adequate amount of dentin color into the immersion container
- Apply 1 or 2 coats of the incisal liquid.
- Place the ZrO_2 framework (with/without prior application of incisal fluid) into the immersion container with the dentin color for approximately 15 - 60 seconds

- After this time, remove the framework from the liquid using the plastic forceps and carefully dab off any excess liquid with a clean paper cloth
- Immediately after staining, place the ZrO₂ framework on a clean, nonabsorbent and temperature-resistant surface (e.g. sinter plate) and dry in a suitable oven for approx. 10 - 30 minutes at approx. 80 - 100 °C (do not use an oven for foodstuffs!).
- After drying, sinter the framework in accordance with the instructions for the framework material in question. Please note the separate sintering instructions at www.merz-dental.de

Storage

The coloring solutions should be stored in the original container, tightly closed, at a maximum of 25 °C / 77 °F.

Disposal

Hazardous waste, disposal according to legal regulations. Non-contaminated and completely empty packages can be recycled.

Danger and safety instructions

H315 Causes skin irritation.

H318 Causes serious eye damage.

H335 May cause respiratory irritation

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P403+P233 Store in a well-ventilated place. Keep container tightly closed.

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from www.merz-dental.de

The product properties are based on compliance with and observation of these instructions for use.

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Merz Dental GmbH