# PremEco® Line

## Alginatisolierung

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- |                                      |   |
|--------------------------------------|---|
| <b>(en)</b> Instructions for use     | <b>(lt)</b> Naudojimo instrukcija       |
| <b>(fr)</b> Notice d'utilisation     | <b>(lv)</b> Lietošanas informācija      |
| <b>(it)</b> Istruzioni per l'uso     | <b>(et)</b> Kasutusjuhend               |
| <b>(es)</b> Instrucciones de uso     | <b>(hu)</b> Használati utasítás         |
| <b>(pt)</b> Instruções de utilização | <b>(cs)</b> Návod k použití             |
| <b>(pl)</b> Instrukcja użycia        | <b>(sl)</b> Navodila za uporabo         |
| <b>(nl)</b> Gebruiksaanwijzing       | <b>(sk)</b> Návod na používanie         |
| <b>(sv)</b> Bruksanvisning           | <b>(el)</b> Οδηγίες χρήσης              |
| <b>(da)</b> Brugsanvisningen         | <b>(tr)</b> Kullanım kılavuzu           |
| <b>(no)</b> Bruksanvisningen         | <b>(uk)</b> Інструкція для застосування |
| <b>(fi)</b> Käyttöohje               |   |

# PremEco® Line

## Alginatisolierung

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Stand der Information 2013-05

# PremEco<sup>®</sup> Line Alginate isolating solution

*Special isolating solution to use when processing heat-curing and cold-curing polymers. Thanks to its optimum consistency it can be easily applied, absorbs easily into plaster and isolates well.*

**(en) Instructions for use, please read carefully!**

## **Areas of Application**

Isolation for all completion work, relines, repairs and additions to partial and full dentures, saddles of model cast dentures, removable saddles of combined dental restorations, immediate and temporary dentures.

## **Composition**

PremEco<sup>®</sup> Line Alginate isolating solution is a formaldehyde-free mixture of water, alginate and preservatives.

## **Dosage and Method of Administration**

- Clean off, separating agent and grease from models and moulds and wet thoroughly with water.
- Allow the plaster surface to dry briefly.
- After boiling out allow the models to cool until water vapour is no longer released.
- Fill the required amount of the insulator into a container.
- Apply the alginate isolating solution once or twice thinly using a clean, soft brush.
- Never dip the brush directly into the bottle containing the alginate insulation solution.
- In order to prevent contamination of the material, do not return any residual quantities back into the bottle.
- Clean used brushes under running water.
- Do not apply to clasps framework parts and teeth.
- Avoid excess solution in grooves, furrows and folds.
- Rinse the isolated model carefully under running water.
- Satisfactory isolation can be identified by a uniform gloss on the surface.

PremEco® Line Alginate isolating solution is a product from the PremEco® Line fabrication system.

*For further information please visit our website at [www.merz-dental.de](http://www.merz-dental.de).*

### **Special Notes**

- To prevent contamination, carefully close the PremEco® Alginate isolating solution bottle after use.
- Store in a dry place at ambient temperature; avoid direct exposure to sunlight.
- Do not use the product after the expiry date.
- Indicate the LOT No. in all processes requiring identification of the material.

**The product properties are based on compliance with and observation of these instructions for use.**

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**Merz Dental GmbH**