# PremEco® Line Prothetik Color System

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# PremEco® Line Prosthetic Color System

Cold-curing, intensively pigmented MMA (methyl methacrylate) based resin

**(en)** Instructions for use, please read carefully!

**(usa)** **Caution:** Federal (USA) law restricts this device to sale by or on the order of a licensed dental practitioner.

## Intended purpose

For the individual colour design of artificial gums in denture fabrication. Standard and intensive colours enable the creation of demanding aesthetic solutions both for new fabrications and for subsequent prosthetic colouration.

## Classification according to MDD 93/42/EEC and MDR (EU) 2017/745

Class IIa medical device for long-term contact with intact mucous membranes.

## Patient target groups

All genders and age groups

## Intended users

Dental professionals

## Indication

- Total or partial tooth loss

## Contraindication

- Hypersensitivity to one of the components

## Composition

PremEco® Line Prosthetic Color System powder contains

- PMMA (polymer and copolymer)
- barbituric acid catalyst system
- organic colourants
- inorganic pigments

## Dosage and application

- Start adding the colours once all conventional fabrication steps have been completed and flask and models are ready for casting.
- Use PremEco® Line casting resin monomer for working.
- The colours can be freely mixed together.
- Application is done using the brush technique with a natural bristle brush and is directly applied onto the gel at the desired location in the flask.
- Select consistency so that the resin can swell to a plastic mass. It should be stable after application so that it does not flow to undesired positions.
- Wet with monomer if there is accelerated drying (white surface).
- After application of the entire amount, re-wet the surface with some monomer.
- Reposition the model in the flask or in the gel.
- Close the flask and fill the PremEco® Line casting resin.
- Curing in the pressure pot for 30 minutes at approximately 2 - 2.5 bar and 45 °C / 113 °F

*PremEco® Line Prosthetic Color System is a product of the PremEco® Line fabrication system.  
For more information see [www.merz-dental.de](http://www.merz-dental.de).*

## **Trimming**

To prevent inaccurate fitting after polymerisation, avoid excessive heat development during trimming and polishing.

## **Restoration**

Dentures that have been individualised with the PremEco® Line Prosthetic Color System can be restored and supplemented at any time with all standard MMA-based cold-curing polymers. Following roughening of the surface (sanding / blasting / grinding), further conditioning is recommended with a similar MMA-based bonding agent.

## **Storage**

- Store dry and cool
- Avoid direct sunlight

## **Notes**

- Close all powder and liquid containers tightly after use.
- Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery.
- Processing frameworks may generate dust which can lead to mechanical irritation of the eyes and airways. Therefore, always ensure that the extraction system on your milling machine and at the workplace is working properly for individual postprocessing and that you are using personal protective equipment.
- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- IF IN CONTACT WITH SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/ shower.
- IF INHALED: Move the person to fresh air and keep comfortable for breathing.
- For more information on processing acrylics and reducing residual monomer content, please visit [www.merz-dental.de/service](http://www.merz-dental.de/service)
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

## **Disposal**

- The fully polymerised components of PremEco® Line Prosthetic Color System are disposed of as residual waste.
- Disposal of nonpolymerised product residues according to legal regulations.
- Do not allow entry into the sewage system or waterways.

## **The product properties are based on compliance with and observance of these instructions for use.**

For detailed information please refer to the relevant Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de).

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**Merz Dental GmbH**