PremEco® Line

Präzisions

Gel NF

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- (en) Instructions for use
- (fr) Notice d'utilisation
- (it) Istruzioni per l'uso
- (es) Instrucciones de uso
- (pt) Instruções de utilização
- (pl) Instrukcja użycia
- (nl) Gebruiksaanwijzing
- (sv) Bruksanvisning
- (da) Brugsanvisningen
- (hu) Használati utasítás
- (sk) Návod na používanie
- (el) Οδηγίες χρήσης
- (uk) Інструкція для застосування

PremEco® Line Präzisionsgel NF

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PremEco® Line NF precision gel

Agar-agar base

(en) Instructions for use, please read carefully!

Intended purpose

Particularly suited for processing acrylic dentures in the pouring technique, for the construction of investment models for partial dentures and for the duplication of plaster models.

Composition

PremEco® Line NF precision gel contains

- water, glycerine, hydrocolloid
- colorant, odorant and preservatives

Dosage and application

- The gel supplied in the container is ready for use.
- Cut the solidified gel into smaller pieces before putting it into the duplicating machine to allow faster melting.
- Melting temperature 93 - 95 °C
- Processing temperature between approx. 50 °C
- The gel can be re-used up to 10 times depending on the contamination.
- If the gel has already been used, remove residues and impurities before re-melting.
- **Note:** Do not use PremEco® Line Precision Gel NF, which was used for phosphate denture investment material, for plaster or plaster-based investment material. Disturbed setting.

It is advisable to add approx. 5 % water with each melting process, as water can escape through evaporation.

Note: When using new gel for the first time, cut it up and place into the duplicating machine without adding water.

Melting in the microwave

- Melt approximately 350 g gel cut into small pieces with 20 ml water in a suitable container in the microwave at 800 W for approx. 2 - 3 minutes under observation.
- The gel must not boil, if necessary interrupt the melting process and stir the gel briefly.
- After melting the gel, allow it to cool down to 50 °C while stirring.

PremEco® Line NF precision gel is a product of the PremEco® Line production system.

For more information see www.merz-dental.de

Model preparation

Use temperature-resistant lay-on/block-out waxes. Soak the plaster models in water. Clean the plaster models to be duplicated and place them in warm water at 40 - 45 °C for 30 minutes with the teeth facing down.

Duplicating

Gently dab off moisture from the soaked parts and place the model in the center of the flask. The melting and cooling process in the duplicating machine should take place with constant stirring. Temperature of the gel during the melting process (93 - 95 °C) and during processing (approx. 50°C (± 1°C)). Check temperatures occasionally with an external thermometer, as only exact temperature compliance leads to optimal results.

Let the gel run into the flask.

Leave the flask to stand for 15 minutes and then place it in a container filled with approx. 2 cm deep water at approx. 18 - 25 °C. Total setting time until removal approx. 30 minutes.

Caution: When using plastic flasks, up to 30% longer solidification times are required.

The duplication can then be carefully removed in parallel. Without a long holding time (max. 1 hour), we recommend that you continue working uninterrupted.

After duplicating, PremEco® Line Precision Gel NF has a smooth, abrasion-resistant surface and can be used without further treatment.

Re-use:

PremEco® Line Precision Gel NF can be melted several times (See processing instructions). Proper processing and correct storage extends the shelf life.

Special notes

- Always store new and used material separately to prevent contamination of new material.
- After approx. the tenth melting process, the gel is used up and loses its good properties. It can then be disposed of with household waste.
- Close the container of PremEco® Line Precision Gel NF carefully after use in order to prevent the gel from drying out.
- Store dry and at room temperature, avoid direct sunlight.
- Do not use the material after the expiry date.
- Record the LOT number with every process that requires identification of the material.

The product properties are based on compliance with the instructions for use.

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Merz Dental GmbH