# Promolux® HIGH IMPACT

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# Promolux® HIGH IMPACT

Heat-curing polymer

 **Instructions for use, please read carefully!**

## Intended purpose

Partial / total prostheses

## Applicability

Hybrid dentures, Overdentures, Combined dentures, Model casting

## Classification according to MDD 93/42/EEC and MDR (EU) 2017/745

Class IIa medical device for long-term contact with intact mucous membranes.

## Patient target groups

From permanent dentition

## Intended users

Dental professionals

## Indication

- Total or partial tooth loss

## Contraindication

- Hypersensitivity to one of the components

## Composition

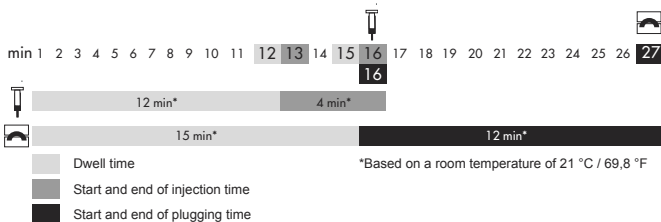
Promolux® High Impact powder contains	Promolux® High Impact liquid contains
- PMMA copolymer	- MMA (methylmethacrylate)
- dibenzoyl peroxide	- dimethacrylate
- organic colorants	
- inorganic pigments	

## Can be used for the following manufacturing processes

-  Injection process as specified by the manufacturer
-  Conventional flask technique (injection and tamping-pressing processes)

## Dosage and application

- Mixing ratio 27,5 g powder : 11 ml liquid
- Use class IV plaster for making casts and investment where possible.
- Use standard alginate isolation to separate plaster from acrylic.
- Do not compress the powder when using dosage aids
- Position monomer and spread polymer by steady tapping.
- Mix well with a spatula for 30 seconds and allow to stand covered.
- The flask should be at a temperature of < 30 °C / 86 °F during the plugging and injection process.

## Polymerisation process

1. Heat flask in the water bath from room temperature to 70 °C / 158 °F.
2. Hold at 70 °C / 158 °F for 30 to 60 minutes.
3. Then heat up to 100 °C / 212 °F.
4. Hold at 100 °C / 212 °F for 30 minutes.
5. Allow to cool slowly in the water bath to room temperature.
6. Do not deflask until it has reached room temperature.

## Trimming

To prevent inaccurate fitting after polymerisation, avoid excessive heat development during trimming and polishing.

## Bond

- Ensure that no impurities such as wax or isolating agent are present.
- Roughen the basal surfaces of the acrylic teeth before bonding.
- Mechanical retention is recommended.
- Adequate mechanical retention is essential for ceramic teeth.
- Use artConnect PLUS bonding agent to support bonding.

## Restoration

Constructions produced from Promolux® High Impact can be restored, supplemented or lined at any time with MMA-based cold-curing polymers, such as Weropress® / LT in the matching shade. Following roughening of the surface (sanding / blasting / grinding), further conditioning is recommended, preferably with artConnect PLUS or a similar MMA-based bonding agent.

## Storage

- Store in a dry and cool area
- Avoid direct sunlight

## Notes

- Close all powder and liquid containers tightly after use.
- Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery.

- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- The residual monomer content after polymerisation of Promolux® High Impact is < 1.8 %.
- Processing frameworks may generate dust which can lead to mechanical irritation of the eyes and airways. Therefore, always ensure that the extraction system on your milling machine and at the workplace is working properly for individual post-processing and that you are using personal protective equipment.
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

### **Disposal**

- The fully polymerized components of Promolux® High Impact are disposed of as residual waste.
- Disposal of nonpolymerized product residues according to legal regulations.
- Do not allow entry into the sewage system or waterways.

### **Danger and safety instruction Promolux® High Impact Monomer**

H225 Highly flammable liquid and vapour.

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P280 Wear protective gloves and eye/face protection.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.

No smoking.

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

P403+P235 Store in a well-ventilated place. Keep cool.

### **Classification**

Promolux® High Impact conforms to EN ISO 20795-1 Typ 1 Class 1 **and meets the requirements for material with increased impact strength.**

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de).

**The product properties are based on compliance with and observation the instructions for use.**

Date of information 2025-05

# Promolux® HIGH IMPACT

Heat-curing polymer

**USA** Instructions for use, please read carefully!

**Caution:** Federal law restricts this device to sale by or on the order of a licensed dental practitioner.

## Intended purpose

Partial / total prostheses

## Applicability

Hybrid dentures, Overdentures, Combined dentures, Model casting

## Classification according to MDD 93/42/EEC and MDR (EU) 2017/745

Class IIa medical device for long-term contact with intact mucous membranes.

## Patient target groups

From permanent dentition

## Intended users

Dental professionals

## Composition

Promolux® High Impact powder contains

- PMMA copolymer
- dibenzoyl peroxide
- organic colorants
- inorganic pigments

Promolux® High Impactliquid contains

- MMA (methylmethacrylate)
- dimethacrylate

## Can be used for the following manufacturing processes

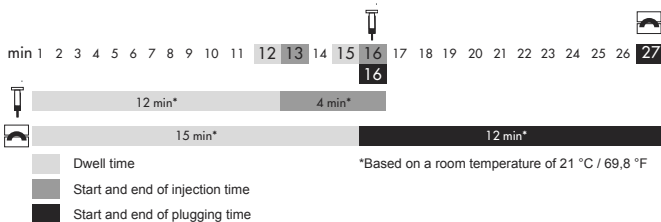
Injection process as specified by the manufacturer

Conventional flask technique

## Dosage and application

- Mixing ratio 27,5 g powder : 11 ml liquid
- Use class IV plaster for making casts and investment where possible.
- Use standard alginate isolation to separate plaster from acrylic.
- Do not compress the powder when using dosage aids.
- Position monomer and spread polymer by steady tapping.
- Mix well with a spatula for 30 seconds and allow to stand covered.
- The flask should be at a temperature of < 30 °C / 86 °F during the plugging and injection process.

## Polymerisation process

1. Heat flask in the water bath from room temperature to 70 °C / 158 °F.
2. Hold at 70 °C / 158 °F for 30 to 60 minutes.
3. Then heat up to 100 °C / 212 °F.
4. Hold at 100 °C / 212 °F for 30 minutes.
5. Allow to cool slowly in the water bath to room temperature.
6. Do not deflask until it has reached room temperature.

## Trimming

To prevent inaccurate fitting after polymerisation, avoid excessive heat development during trimming and polishing.

## Bond

- Roughen the basal surfaces of the acrylic teeth before bonding.
- Ensure that no impurities such as wax or isolating agent are present.
- Mechanical retention is recommended.
- Adequate mechanical retention is essential for ceramic teeth.

## Restoration

Constructions produced from Promolux® High Impact can be restored, supplemented or lined at any time with MMA-based cold-curing polymers, such as Weropress®.

## Special notes

- Close all powder and liquid containers tightly after use.
- Store in a dry and cool area away from direct sunlight.
- Take precautionary measures against static discharges.
- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- The residual monomer content after polymerisation of Promolux® High Impact is < 1,8 %.
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

## **Disposal information**

- The fully polymerized components of Promolux® High Impact are disposed of as residual waste.
- Disposal of nonpolymerized product residues according to legal regulations.
- Do not allow entry into the sewage system or waterways.

## **Danger and safety instruction Promolux® High Impact Monomer**

Highly flammable liquid and vapor

Causes skin irritation

May cause an allergic skin reaction

May cause respiratory irritation

Return contents/container to the point of sale or to a special waste collection point.

Keep away from heat/sparks/open flames/hot surfaces. - No smoking.

Keep container tightly closed.

Ground/bond container and receiving equipment.

Use explosion-proof electrical/ventilating/lighting equipment.

Use only non-sparking tools.

Take precautionary measures against static discharge.

Avoid breathing dust/fume/gas/mist/vapors/spray.

Use only outdoors or in a well-ventilated area.

Contaminated work clothing must not be allowed out of the workplace.

Wear protective gloves and eye protection/face protection.

If on skin (or hair): Take off immediately all contaminated clothing.

Rinse skin with water/shower.

If skin irritation or rash occurs: Get medical advice/attention.

Take off contaminated clothing and wash it before reuse.

If inhaled: Remove person to fresh air and keep comfortable for breathing.

Call a poison center/doctor if you feel unwell.

Keep container tightly closed.

Store in a well-ventilated place. Keep cool.

Store locked up.

## **Classification**

Promolux® High Impact conforms to EN ISO 20795-1 Typ 1 Class 1 **and meets the requirements for material with increased impact strength.**

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de).

**The product properties are based on compliance with and observation the instructions for use.**

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**Merz Dental GmbH**