Weropress® / Weropress® LT

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Weropress® / Weropress® LT

Cold-curing polymer

(en) Instructions for use, please read carefully!

Intended purpose

Cold-curing, colour-stable, methylmethacrylate-based resin for fabricating partial and total prostheses using injection, tamping-pressing and casting processes.

Classification according to MDD 93/42/EEC and MDR (EU) 2017/745

Class IIa medical device for long-term contact with intact mucous membranes.

Patient target groups

All genders and age groups

Intended users

Dental professionals

Indication

- Total or partial tooth loss
- Abnormal jaw function

Contraindication

- Hypersensitivity to one of the components

Composition

Weropress® Polymer	Weropress® Monomer	Weropress® LT Monomer
PMMA (polymer und copolymer)	MMA (methylmethacrylate)	MMA (methylmethacrylate)
barbituric acid catalyst system	dimethacrylate	dimethacrylate
organic colourants	barbituric acid catalyst system	ethermethacrylate
inorganic colourants		barbituric acid catalyst system

Injection process

- Proceed as specified by the device manufacturer
- The temperature of the plaster in the flask should be 35 - 40 °C / 95 - 104 °F to promote polymerisation.

Press process

- Flask technique (injection and tamping-pressing processes)
- The temperature of the plaster in the flask should be 35 - 40 °C / 95 - 104 °F to promote polymerisation.
- Complete the press process promptly under the hydraulic press within 1 minute.

Casting process

- Investment with plaster, silicone or hydrocolloid (gel) depending on the process.
- Polymerisation is conducted at 2 to 2.5 bar pressure in water at 45 °C / 113 °F after appropriate dwell time.

Dosage and application

- See table for mixing ratios and processing times depending on the processing method of the acrylic.
- Use class IV plaster for making casts and investment where possible.
- Always wet casts sufficiently for 5 to 15 minutes depending on the degree of dryness.
- Use standard alginate isolation to separate plaster from acrylic.
- Do not compress the powder when using dosage aids.


Times and quantities

	Injection process	Tamping-pressing process	Casting process
Mixing ratio*	30 g : 15 ml	30 g : 15 ml	30 g : 21 ml
Polymerisation temperature	Residual heat in the flask	Residual heat in the flask	45 °C / 113 °F
Polymerisation time	15 min	15 min	15 min ¹ / 30 min ²
Pressure required during polymerisation	By device type	Pressure in the flask must be maintained during polymerisation	2-2,5 bar

Weropress®

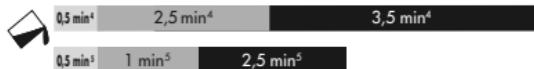Weropress® LT

Mixing time

Dwell time

Processing time (pourable consistency)
(Start and end of injection time/tamping time)

¹ in matrix technique, plaster, putty

² in flask duplication technique, duplication gel, duplication silicone

³ see also dosage aid REF 1090194

⁴ Based on a room temperature of 21 °C / 69,8 °F

⁵ Based on a room temperature of 30 °C / 86 °F

Trimming

To prevent inaccurate fitting after polymerisation, avoid excessive heat development during trimming and polishing.

Bond

- Roughen the basal surfaces of the acrylic teeth before bonding.
- Ensure that no impurities such as wax or isolating agent are present.
- Mechanical retention is recommended.
- Adequate mechanical retention is essential for ceramic teeth.

Restoration

Constructions produced from Weropress® / LT can be restored, supplemented or lined at any time with MMA-based cold-curing polymers, such as Weropress® / LT, Combipress N/LM or artDentine PLUS / artEnamel PLUS in the matching shade. Following roughening of the surface (sanding / blasting / grinding), further conditioning is recommended, preferably with artConnect PLUS or a similar MMA-based bonding agent.

Disposal

- The fully polymerized components of Weropress® / LT are disposed of as residual waste.
- Disposal of nonpolymerized product residues according to legal regulations.
- Do not allow entry into the sewage system or waterways.

Notes

- Close all powder and liquid containers tightly after use.
- Store in a dry and cool area away from direct sunlight.
- Changes to the construction through use, e.g. due to fissures, cracks, clouding or significant signs of abrasion, necessitate a repair or refabrication. Monitoring should be carried out with regular assessment of the denture at the follow-up presentation in the dental surgery.
- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- The residual monomer content after polymerisation of Weropress® / LT is < 4,5% (EN ISO 20795-1).
- For more information on processing acrylics and reducing residual monomer content, please visit www.merz-dental.de/service
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

Warnings

Processing frameworks may generate dust which can lead to mechanical irritation of the eyes and airways. Therefore, always ensure that the extraction system on your milling machine and at the workplace is working properly for individual post-processing and that you are using personal protective equipment.

Danger and safety instruction Weropress® & Weropress® LT Monomer

H225 Highly flammable liquid and vapour.

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

P233 Keep container tightly closed.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P262 Do not get in eyes, on skin, or on clothing.

P210 Keep away from heat. No Smoking.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

Danger and safety instructions Weropress® Polymer

EUH208 Contains dibenzoyl peroxide; benzoyl peroxide, methyl 2-methylprop-2-enoate; methyl 2-methylpropenoate; methyl methacrylate. May produce an allergic reaction.

Classification

Weropress® / LT conforms to EN ISO 20795-1 Type 2 Class 1.

The product properties are based on compliance with and observation the instructions for use.

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from www.merz-dental.de.

Date of information 2022-12

Weropress®

Cold-curing polymer

usa can Instructions for use, please read carefully!

**Caution: Federal law restricts this device to sale by or
on the order of a licensed dental practitioner.**

Intended purpose

Cold-curing, colour-stable, methylmethacrylate-based resin for fabricating partial and total prostheses using injection, tamping-pressing and casting processes.

Classification according to MDD 93/42/EEC and MDR (EU) 2017/745

Class Ila medical device for long-term contact with intact mucous membranes.

Patient target groups

All genders and age groups

Intended users

Dental professionals

Composition

Weropress® powder contains

- PMMA (polymer and copolymer)
- barbituric acid catalyst system
- organic colorants
- inorganic pigments

Weropress® liquid contains

- MMA (methylmethacrylate)
- dimethacrylate
- barbituric acid catalyst system

Dosage and application

- See table for mixing ratios and processing times depending on the processing method of the acrylic.
- Always wet casts sufficiently for 5 to 15 minutes depending on the degree of dryness.
- Use standard alginate isolation to separate plaster from acrylic.
- Do not compress the powder when using dosage aids.

Injection process

- Proceed as specified by the device manufacturer
- Use class IV plaster for making casts and investment where possible.
- The temperature of the plaster in the flask should be 35 - 40 °C / 95 - 104 °F to promote polymerisation.

Press process

- Use class IV plaster for making casts and investment where possible.
- The temperature of the plaster in the flask should be 35 - 40 °C / 95 - 104 °F to promote polymerisation.
- Complete the press process promptly under the hydraulic press within 1 minute.

Casting process

- Use class IV plaster for making casts where possible.
- Investment with plaster, silicone or hydrocolloid (gel) depending on the process.
- Polymerisation is conducted at 2 to 2.5 bar pressure in water at 45 °C / 113 °F after appropriate dwell time.

Times and quantities

	Injection process	Tamping-pressing process	Casting process
Mixing ratio*	30 g : 15 ml	30 g : 15 ml	30 g : 21 ml
Polymerisation temperature	Residual heat in the flask	Residual heat in the flask	45 °C / 113 °F
Polymerisation time	15 min	15 min	15 min ¹ / 30 min ²
Pressure required during polymerisation	By device type	Pressure in the flask must be maintained during polymerisation	2-2,5 bar

Mixing time

Dwell time

Processing time (pourable consistency)
(Start and end of injection time/tamping time)

¹ in matrix technique, plaster, putty

² in flask duplication technique, duplication gel, duplication silicone

*see also dosage aid **REF 1090194**

**Based on a room temperature of 21 °C / 69,8 °F

Trimming

To prevent inaccurate fitting after polymerisation, avoid excessive heat development during trimming and polishing.

Bond

- Roughen the basal surfaces of the acrylic teeth before bonding.
- Ensure that no impurities such as wax or isolating agent are present.
- Mechanical retention is recommended.
- Adequate mechanical retention is essential for ceramic teeth.

Restoration

Constructions produced from Weropress® / LT can be restored, supplemented or lined at any time with MMA-based cold-curing polymers, such as Weropress® / LT in the matching shade. Following roughening of the surface (sanding / blasting / grinding), further conditioning is recommended with an MMA-based bonding agent.

Notes

- Close all powder and liquid containers tightly after use.
- Store in a dry and cool area away from direct sunlight.
- Take precautionary measures against static discharges.
- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- The residual monomer content after polymerisation of Weropress® is < 4,5% (EN ISO 20795-1).
- For more information on processing acrylics and reducing residual monomer content, please visit www.merz-dental.de/service
- All serious incidents occurring with users and / or patients in connection with this product must be reported to Merz Dental, as well as to the German Federal Institute for Drugs and Medical Devices (BfArM), or the respective competent authority of the member state in which the user and / or patient is resident.

Disposal information

Disposal of contents/container in accordance with local/regional/national/international regulations.

Danger and safety instruction Weropress® Monomer

H225 Highly flammable liquid and vapour.

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

P233 Keep container tightly closed.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P262 Do not get in eyes, on skin, or on clothing.

P210 Keep away from heat. No Smoking.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

Danger and safety instructions Weropress® Polymer

EUH208 Contains dibenzoyl peroxide; benzoyl peroxide, methyl 2-methylprop-2-enoate; methyl 2-methylpropenoate; methyl methacrylate. May produce an allergic reaction.

Classification

Weropress® conforms to EN ISO 20795-1 Type 2 Class 1.

The product properties are based on compliance with the instructions for use.

For detailed information see the applicable Material Safety Data Sheet.

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Merz Dental GmbH