## Baltic Denture System

Digital solution for dentures

# BDKEY<sup>®</sup> Plane & BDKEY<sup>®</sup> Fin

- |                                    |                                       |
|------------------------------------|---------------------------------------|
| <b>de</b> Gebrauchsinformation     | <b>it</b> Informacinis lapelis        |
| <b>en</b> Instructions for use     | <b>lv</b> Lietošanas informācija      |
| <b>fr</b> Mode d'emploi            | <b>et</b> Kasutusteave                |
| <b>it</b> Istruzioni per l'uso     | <b>hu</b> Használati tájékoztatás     |
| <b>es</b> Instrucciones de uso     | <b>cs</b> Informace k použití         |
| <b>pt</b> Instruções de utilização | <b>sl</b> Navodila za uporabo         |
| <b>pl</b> Instrukcja użycia        | <b>sk</b> Návod na použitie           |
| <b>nl</b> Gebruiksaanwijzing       | <b>bg</b> Информация за употреба      |
| <b>sv</b> Bruksanvisning           | <b>el</b> Οδηγίες χρήσης              |
| <b>da</b> Brugsanvisning           | <b>tr</b> Kullanım bilgisidir         |
| <b>no</b> Bruksanvisning           | <b>uk</b> Інструкція для застосування |
| <b>fi</b> Käyttöohje               |                                       |

MADE IN GERMANY

# BDKEY® Plane & BDKEY® Fin

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# BDKEY® Plane & BDKEY® Fin

**en** Instructions for use, please read carefully!

## Technical applicability

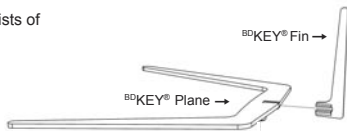
The **BDKEY® Plane & BDKEY® Fin** set serves as an optional tool for visualization of the occlusal plane and centre of face. It is intended for use together with the self-printable **BDKEY® Innerplane**.

**BDKEY® Plane & BDKEY® Fin** consists of

- 1 x **BDKEY® Plane**
- 1 x **BDKEY® Fin**

## Intended users

Dental professionals

## Application instructions

- The **BDKEY® Fin** can be attached at the designated spot, centred on the **BDKEY® Plane**.
- The **BDKEY® Plane, BDKEY® Fin & BDKEY® Innerplane** can only be attached in one direction.
- The **BDKEY® Innerplane** may be optionally stored as a dataset at [www.merz-dental.de](http://www.merz-dental.de).
- The **BDKEY®** in sizes S, M and L may be optionally stored as a dataset at [www.merz-dental.de](http://www.merz-dental.de).

## Storage

Protect against direct sunlight..

The **BDKEY® Plane & BDKEY® Fin** are cleaned and visually inspected before use as follows.

## Cleaning and visual inspection

- Rinse off soiling under running water; take special care given the presence of cavities.
- Cleaning can be carried out with a soft brush.
- No additional cleaning agents should be used.
- After cleaning, the products should be visually inspected for residues or damage.

---

For detailed information please see the applicable Material Safety Data Sheet, available upon request or as download from [www.merz-dental.de](http://www.merz-dental.de)

---

## Disposal

The **BDKEY® Plane & BDKEY® Fin** components are water-insoluble, inactive, do not pose a threat to groundwater and may therefore be disposed of with normal household waste.

**The product properties are based on compliance with the instructions for use**

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**Merz Dental GmbH**