# PremEco® Line

Gießkuvette

## Gebrauchsinformation

- |                                      |                                   |
|--------------------------------------|-----------------------------------|
| <b>en</b> Instructions for use       | <b>lt</b> Informacinis lapelis    |
| <b>fr</b> Mode d'emploi              | <b>lv</b> Lietošanas informācija  |
| <b>it</b> Istruzioni per l'uso       | <b>et</b> Kasutusteave            |
| <b>es</b> Instrucciones de uso       | <b>hu</b> Használati tájékoztatás |
| <b>pt</b> Instruções de utilização   | <b>cs</b> Informace k použití     |
| <b>pl</b> Instrukcja użycia          | <b>sl</b> Navodila za uporabo     |
| <b>nl</b> Gebruiksaanwijzing         | <b>sk</b> Návod na použitie       |
| <b>sv</b> Information för användning | <b>el</b> Οδηγίες χρήσης          |
| <b>da</b> Brugsanvisningen           | <b>tr</b> Kullanım bilgidir       |
| <b>fi</b> Käyttöohje                 |                                   |

# PremEco® Line Gießkuvette (Ø 110 mm)

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# PremEco® Line Casting Flask (Ø 110 mm)

A robust, non-distorting precision duplicating flask for the resin casting technique for fabricating partial and total resin and model cast prostheses, for double-crown and cover-denture prostheses and for bite raisers of all kinds.

## **(en)** Information for Use. Please read carefully.

### Areas of Application

For the rational completion in the field of polymer prosthetics based on gel and hydrocolloid embedding requiring time-saving preparation and finishing after polymerisation.

Suitable for use with methylmethacrylate-based prosthetic basic resins.

### Handling

- Opening and closing the flask using the three closure screws.
- Observe the markings on the side when assembling.
- Agar-agar-based hydrocolloid is recommended for filling the flask.

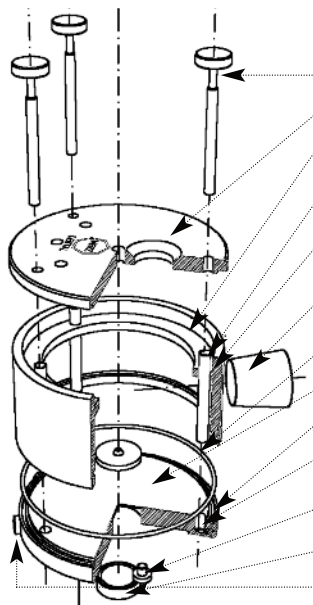
The PremEco® Line Casting Flask is a module from the Karl-Heinz Köhrholz PremEco® Line fabrication system. For further information please visit our Internet site at [www.merz-dental.de](http://www.merz-dental.de).

### Supply Units:

- 1 casting flask, normal (Ø 110 mm) incl. 1 screwdriver and 1 channel cutter
- 1 casting flask, large (Ø 125 mm) incl. 1 screwdriver and 1 channel cutter
- 1 channel cutter

### Art.-No.:

- 1030078
- 1030079
- 1030080

The flask is constructed as shown below:

Three knurled-head screws

Robust flask cover

Stabilizing bar for duplicating gel when opening the flask occlusion protection against polymerisation pressure

Fitted sleeves for exact positioning of cover, upper part and bottom

Markings on the housing and flask bottom facilitate assembly

2 large sealing plugs close the fill openings for resin and provide an adequate shrinkage reservoir

O-ring ensures seal of the flask and thus polymerisation pressure on one side on the prosthesis base

Aluminium flask floor for better temperature conductivity

Groove for O-ring

Fitted bore holes for establishing anti-twist safeguard in the model flask bottom

3 stainless steel adjusting feet for controlled, uniform cooling of the gel

Removable magnet for securing the model

Stainless steel feet ensure a secure position when filling

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**Merz Dental GmbH**