PremEco® Line Prothetik Color System

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Stand der Information 2014-09

PremEco® Line Prothetik Color System

en Instructions for use, please read carefully!

Indications

Cold-curing, intensively pigmented MMA (methyl methacrylate) based resin for customized colouring of the artificial gums for prosthesis fabrication. Standard and intensive colours make it possible to create demanding aesthetic solutions both for new fabrications and for subsequent prosthetic colouration.

Composition

PremEco® Line Prothetik Color System powder contains

- PMMA (polymer and copolymer)
- barbituric acid catalyst system
- organic colorants
- inorganic pigments

Dosage and application

- Start adding the colours once all conventional fabrication steps have been completed and the flask and the models are ready for casting.
- Use PremEco® Line casting resin monomer for working.
- The colours can be freely mixed together.
- Application is done using the brush technique with a natural bristle brush and is adapted directly in the flask at the desired point.
- Select consistency so that the resin can swell to a plastic mass. It should be stable after application so that it does not flow to undesired positions.
- Wet with monomer if there is accelerated drying (white surface).
- After application of the entire amount, re-wet the surface with some monomer.
- Reposition the model in the cuvette or in the gel.
- Close the flask and fill the PremEco® Line casting resin.
- Curing in the pressure pot for 40 minutes at approximately 2 - 2.5 bar and 45 °C / 113 ° F.

PremEco® Line Prothetik Color System is a product of the PremEco® Line production system. *For more information see www.merz-dental.de.*

Processing the acrylic

To reduce the residual monomer content, the finished (sanded) but not polished polymerised workpiece should be stored in water at room temperature for about 48 hours before delivery.

Restoration

Products of PremEco® Line Prothetik Color System can be restored and supplemented at any time with all standard MMA-based cold-curing polymers, such as Weropress® and Combipress N/LM.

Special notes

- Close the PremEco® Line Prosthetic Colour System powder containers tightly after use.
- Store in a dry and cool area away from direct sunlight.
- Take precautionary measures against static discharges.
- Do not use the material after the expiration date.
- Record the LOT number with every process that requires identification of the material.
- IF ON SKIN (or hair): Take off immediately all contaminated clothing.
Rinse skin with water/ shower.
- IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Disposal information

Disposal of contents/container in accordance with local/regional/national/international regulations.

The product properties are based on compliance with the instructions for use.

For detailed information see the applicable Material Safety Data Sheet.

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Merz Dental GmbH