PremEco® Line

- de Alginatisolierung
- en Alginate Insulation
- fr Isolant alginate
- it Isolante alginico
- es Aislante de alginato
- pt Isolamento de alginato
- pl Izolacja alginowa
- nl Alginaatisolatie
- sv Alginatisolering
- no Alginatisolering
- da Alginatisolering
- fi Alginaattieriste
- lt Alginato izoliacija
- lv Algināta izolācijas līdzeklis
- et Alginaatisolatsioon
- hu Alginát szigetelő
- cs Alginátový izolační prostředek
- sl Alginatna izolacija
- sk Alginátová izolačná zmes
- el Μόνωση αλγινικού
- tr İzolasyon maddesi

PremEco® Line

Alginatisolierung

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PremEco® Line

Alginate Insulation

Special insulating medium for use when processing hotcuring and cold-curing polymers. By virtue of its optimum consistency it can be easily applied, absorbs easily into the plaster and insulates well.

(en) Information for Use. Please read carefully.

Areas of Application

Insulation for all completion work, base fillings, repairs and enhancements of partial and total dentures, saddles of model cast dentures, removable saddles of combined dental prostheses, immediate and temporary dentures.

Composition

PremEco® Line Alginate Insulation is a formaldehyde-free mixture of water, alginate and preservative.

Dosage and Method of Administration

- Clean wax, separating medium and fat from the models and the moulds and wet thoroughly with water.
- Allow the plaster surface to dry briefly.
- After boiling allow the models to cool until water vapour no longer is released.
- Fill the required amount of the insulator into a container.
- Apply the alginate solution once or twice thinly using a clean, soft brush.
- Never dip the brush directly into the bottle containing the alginate insulation solution.
- In order to prevent contamination of the material, do not return any residual quantities back into the bottle.
- Clean used brushes under running water.
- Do not wet clamps, apparatus parts and teeth.
- Avoid excess solution in grooves, furrows and overlaps.
- Rinse the insulated model carefully under running water.
- Satisfactory insulation can be identified by a uniform gloss on the surface.

PremEco® Line Alginate Insulation is a product from the PremEco® Line fabrication system.

For further information please visit our Internet site at www.merz-dental.de.

Special Notes

- To prevent contamination, carefully close the PremEco® Alginate Insulation container after use.
- Store in a dry place at ambient temperature; avoid direct exposure to sunlight.
- Do not use the product after the expiry date.
- Indicate the LOT No. in all processes requiring identification of the material.

The product properties are based on compliance with the instructions for use.

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Merz Dental GmbH