



Anycubic Photon Mono Printer Instructions

CreaPRINT TRAY hot-pink

INDIVIDUELLE ABDRUCKLÖFFEL

REF 1084221

PRINTER SETTINGS

Layer Height	0,1 mm
Bottom Layer Count	6
Exposure Time	8 sec.
Bottom Exposure Time	40 sec.
Transition Layer Count	0
Transition Type	linear
Waiting Mode During Printing	
Rest Time Before Lift	2 sec.
Rest Time After Lift	
Rest Time After Retract	
Bottom Lift Distance	
Lifting Distance	
Bottom Retract Distance	
Retract Distance	
Bottom Lift Speed	
Lifting Speed	
Bottom Retract Speed	
Retract Speed	

