



ELEGOO Saturn 3 ultra Printer Instructions



CreaPRINT TRY-IN ivory

EINPROBEKÖRPER / INTERIMSPROTHESEN

REF 1084231

PRINTER SETTINGS

Layer Height	0,05 mm
Bottom Layer Count	5
Exposure Time	8 sec.
Bottom Exposure Time	70 sec.
Transition Layer Count	5
Transition Type	linear
Waiting Mode During Printing	
Rest Time Before Lift	2 sec.
Rest Time After Lift	
Rest Time After Retract	
Bottom Lift Distance	3 mm + 3 mm
Lifting Distance	3 mm + 3 mm
Bottom Retract Distance	3 mm + 3 mm
Retract Distance	3 mm + 3 mm
Bottom Lift Speed	75 & 230 mm / min.
Lifting Speed	50 & 200 mm / min.
Bottom Retract Speed	200 & 50 mm / min.
Retract Speed	200 & 50 mm / min.

