



SHINING 3D AccuFab-L4D Printer Instructions



CreaPRINT TRAY aqua

INDIVIDUELLE ABDRUCKLÖFFEL

REF 1084220



PRINTER SETTINGS HIGH POWER MODE

Layer Counts	4	26	0
Exposure Level	HPM	HPM	HPM
Fill Exposure Time	15,0 sec.	5,0 sec.	5,0 sec.
Boundary Exposure Time	15,0 sec.	5,0 sec.	5,0 sec.
Support Exposure Time	15,0 sec.	5,0 sec.	5,0 sec.
Wait Time on Top	1,0 sec.	2,0 sec.	0 sec.
Wait Time before Exposure	3,5 sec.	3,5 sec.	2,5 sec.
Wait Time after Exposure	0 sec.	0 sec.	0 sec.
Slow Lift Length on z-axis	2,0 mm	2,0 mm	1,0 mm
Slow Lift Speed on z-axis	50 mm/min	72 mm/min	72 mm/min
Quick Lift Length on z-axis	8 mm	8 mm	6,5 mm
Quick Lift Speed on z-axis	180 mm/min	220 mm/min	330 mm/min
Slow Return Length on z-axis	3,0 mm	2,0 mm	1,0 mm
Slow Return Speed on z-axis	80 mm/min	90 mm/min	72 mm/min
Quick Return Length on z-axis	7 mm	8 mm	6,5mm
Quick Return Speed on z-axis	180 mm/min	220 mm/min	330 mm/min
Print Base Time	21 sec.	14 sec.	12 sec.