



Phrozen 8K mini Printer Instructions

M-PRINT Model 2.0 shark-grey

MODELLHERSTELLUNG

REF 1084022 1.000g

PRINTER SETTINGS

Layer Height	0,1 mm
Bottom Layer Count	5
Exposure Time	4,5 sec.
Bottom Exposure Time	25 sec.
Transition Layer Count	10
Transition Type	linear
Waiting Mode During Printing	1,86 sec.
Rest Time Before Lift	0 sec.
Rest Time After Lift	0 sec.
Rest Time After Retract	4 sec.
Bottom Lift Distance	3 mm
Lifting Distance	3 mm
Bottom Retract Distance	3 mm
Retract Distance	3 mm
Bottom Lift Speed	60 mm/min.
Lifting Speed	60 mm/min.
Bottom Retract Speed	100 mm/min.
Retract Speed	100 mm/min.

