

projectAR Installation instructions



Stand 11/2019

Seite 1 von 8


Dear customer,

Thank you for purchasing the **projectAR** multimedia glasses. For a successful licensing, installation and connection of the product with your intraoral scanner, please note the following information.

Register on the website www.my-projectAR.com and enter your license key.

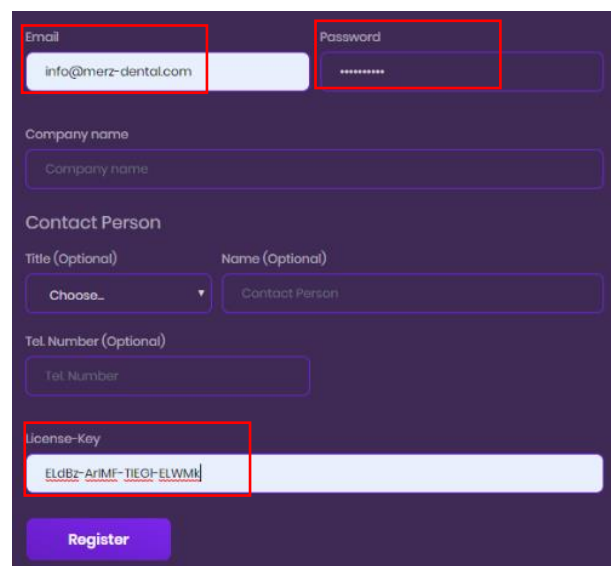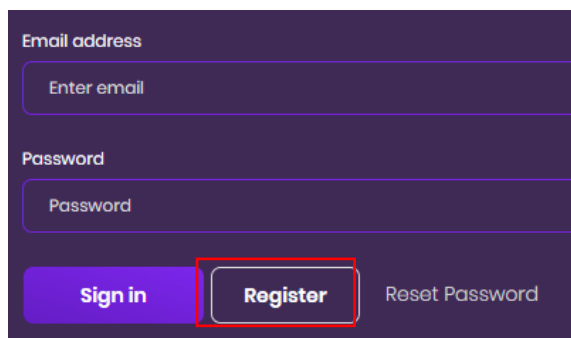

Your license key is now activated and you have access to the **projectAR** portal.

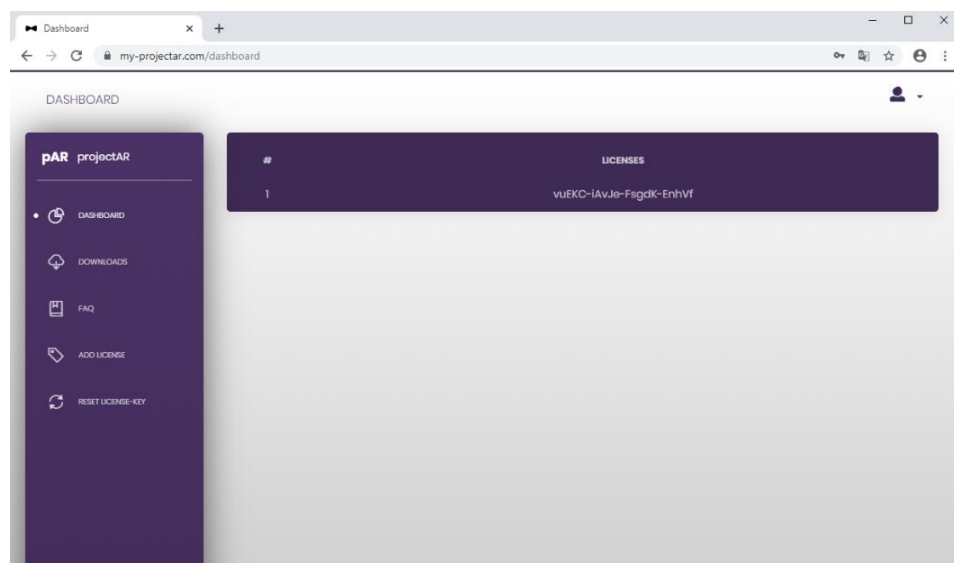


projectAR Installation instructions


Stand 11/2019

Seite 2 von 8

Select >>DOWNLOADS >>Download

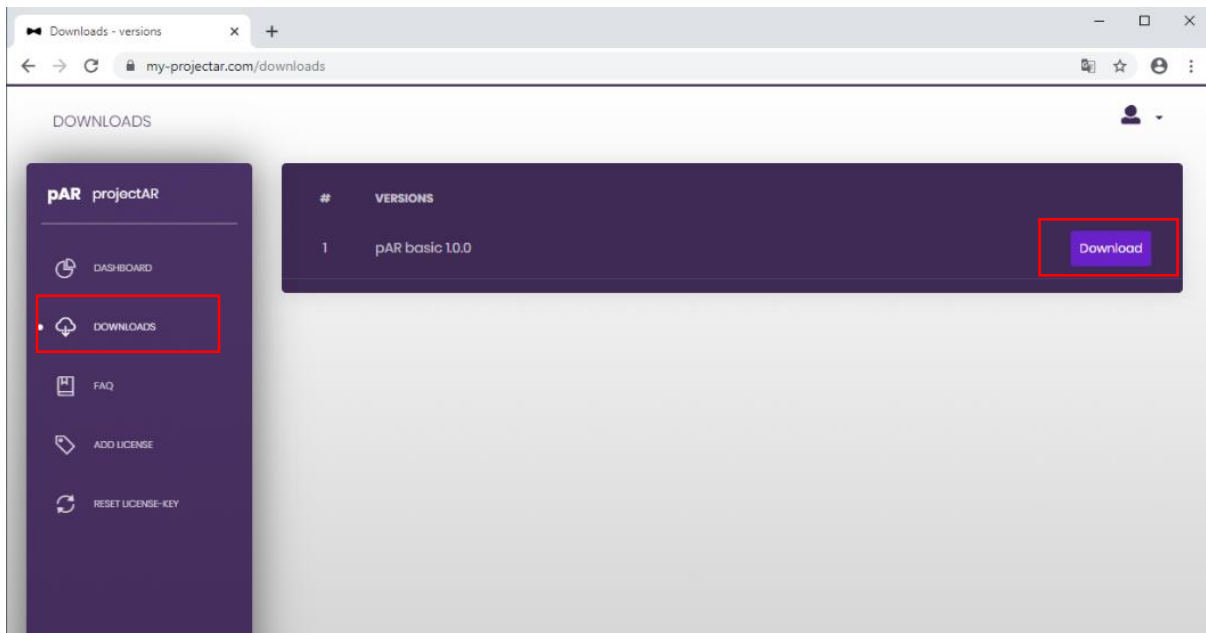

Download the **Bonjour** file to your computer / CAD system.

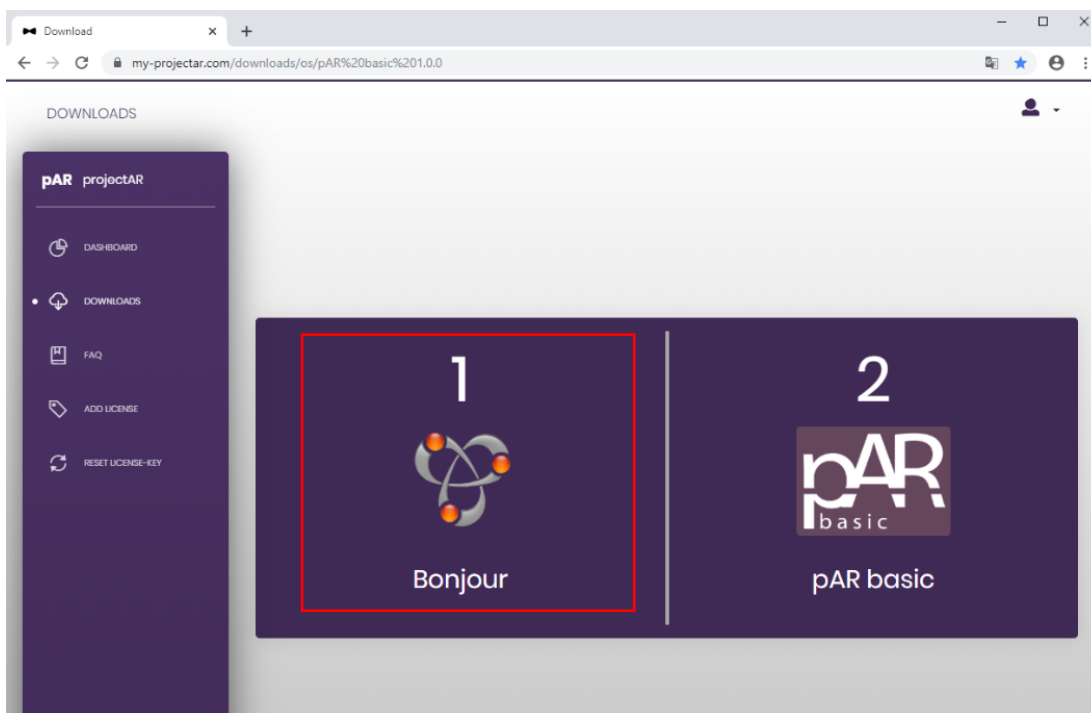

projectAR Installation instructions

Stand 11/2019

Seite 3 von 8

Follow the installation wizard.






projectAR Installation instructions

Stand 11/2019

Seite 4 von 8


Then download and install the **pAR basic** file.

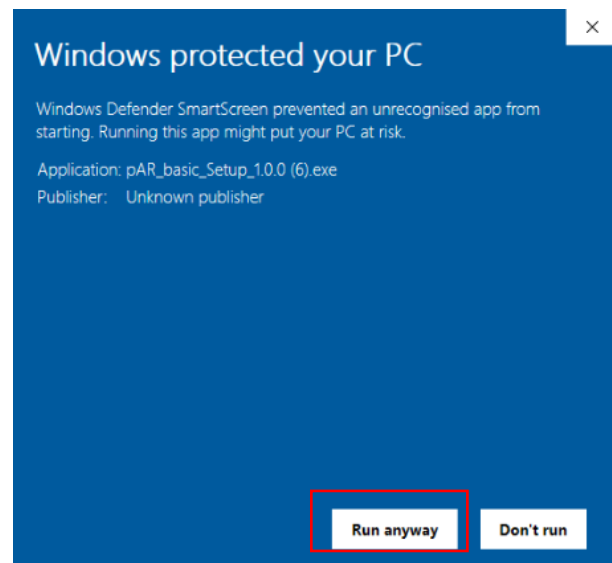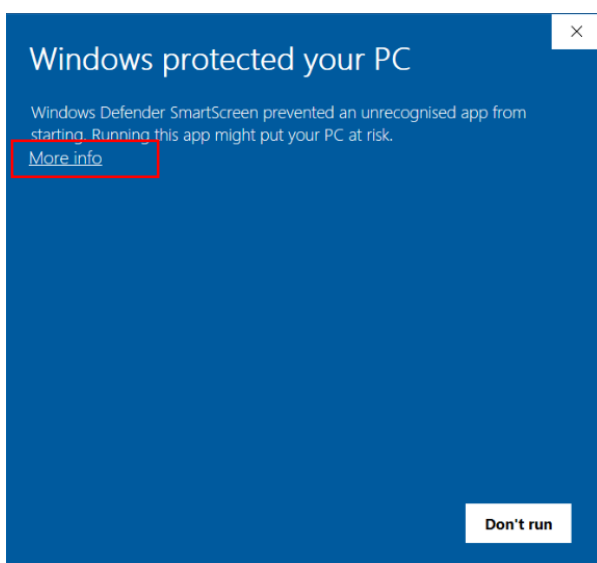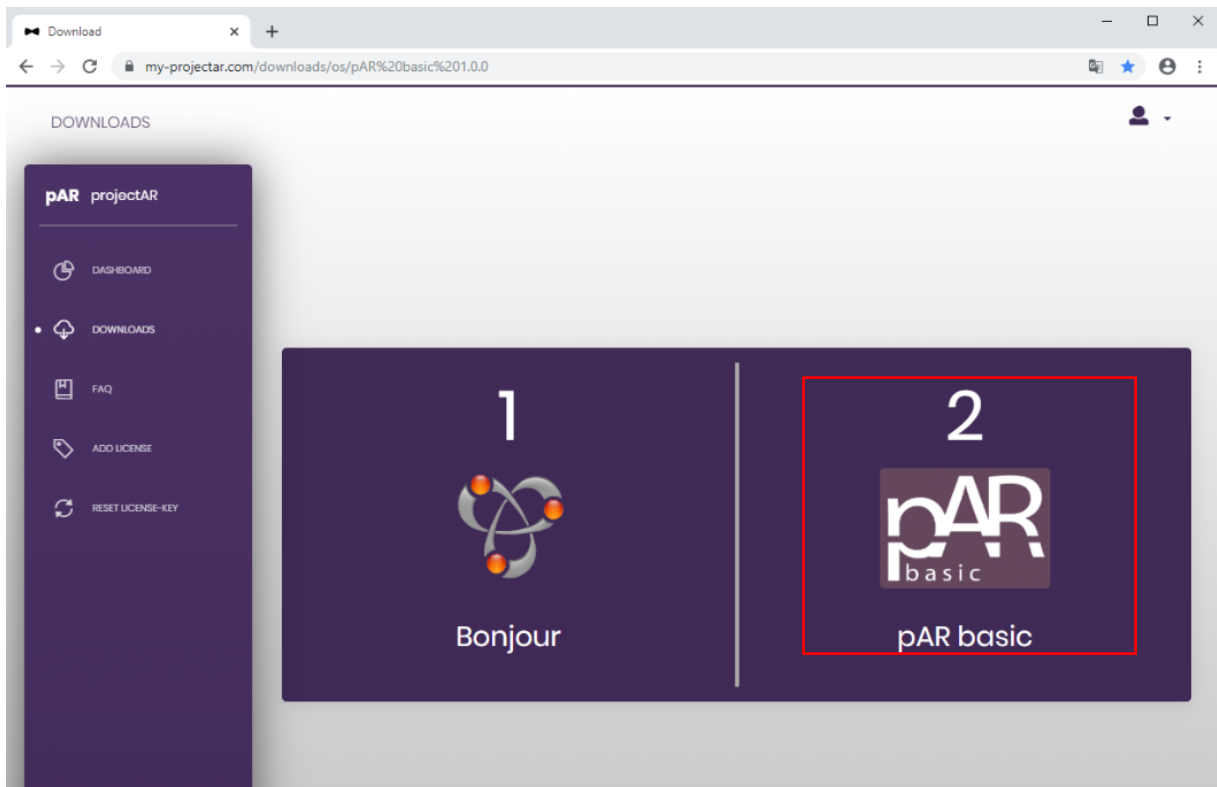


projectAR Installation instructions

Stand 11/2019

Seite 5 von 8

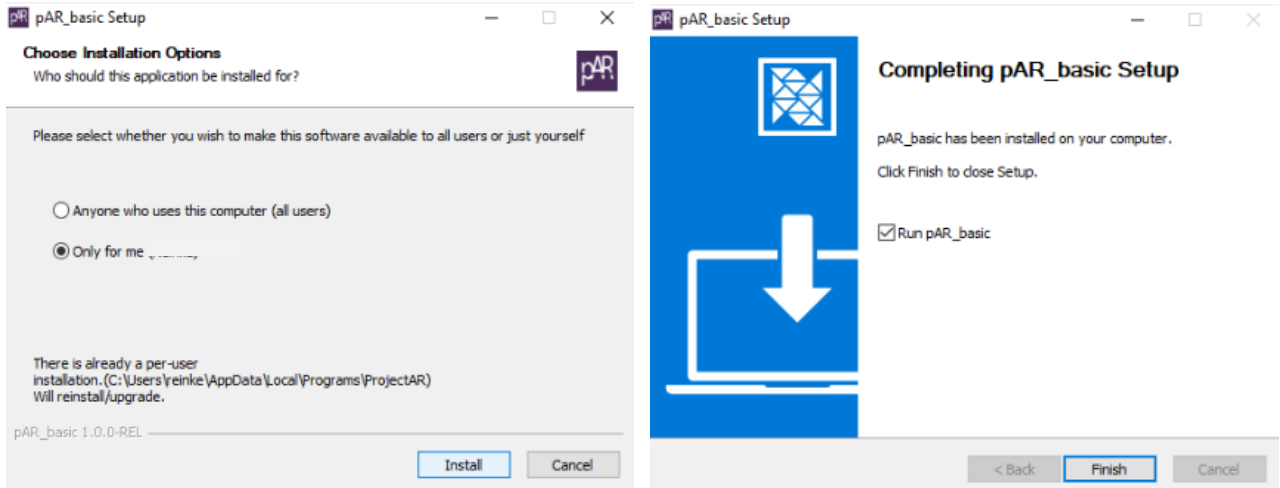



Start the **pAR basic** software and follow the setup wizard. Note that you must re-enter your license key in the software.

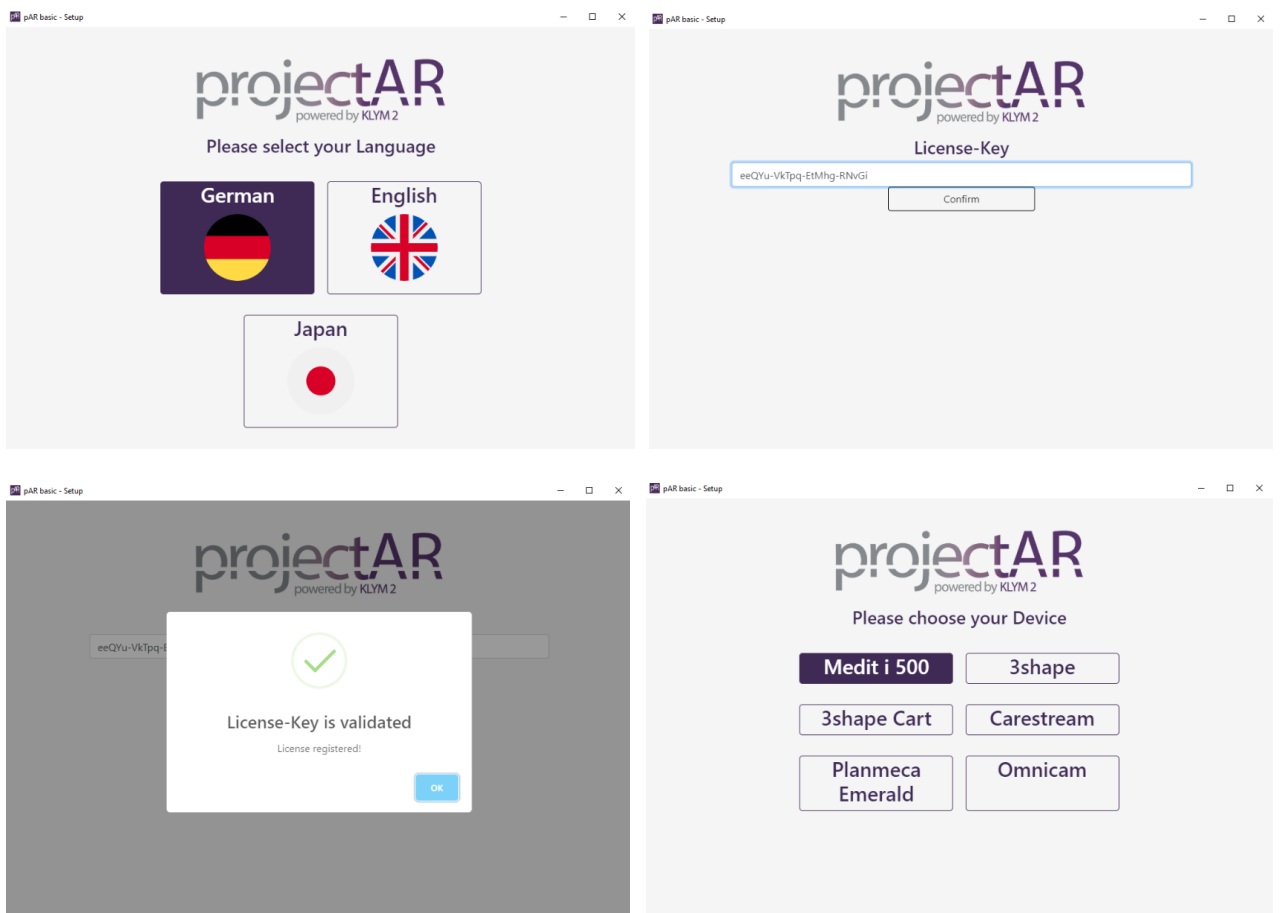

projectAR Installation instructions

Stand 11/2019

Seite 6 von 8

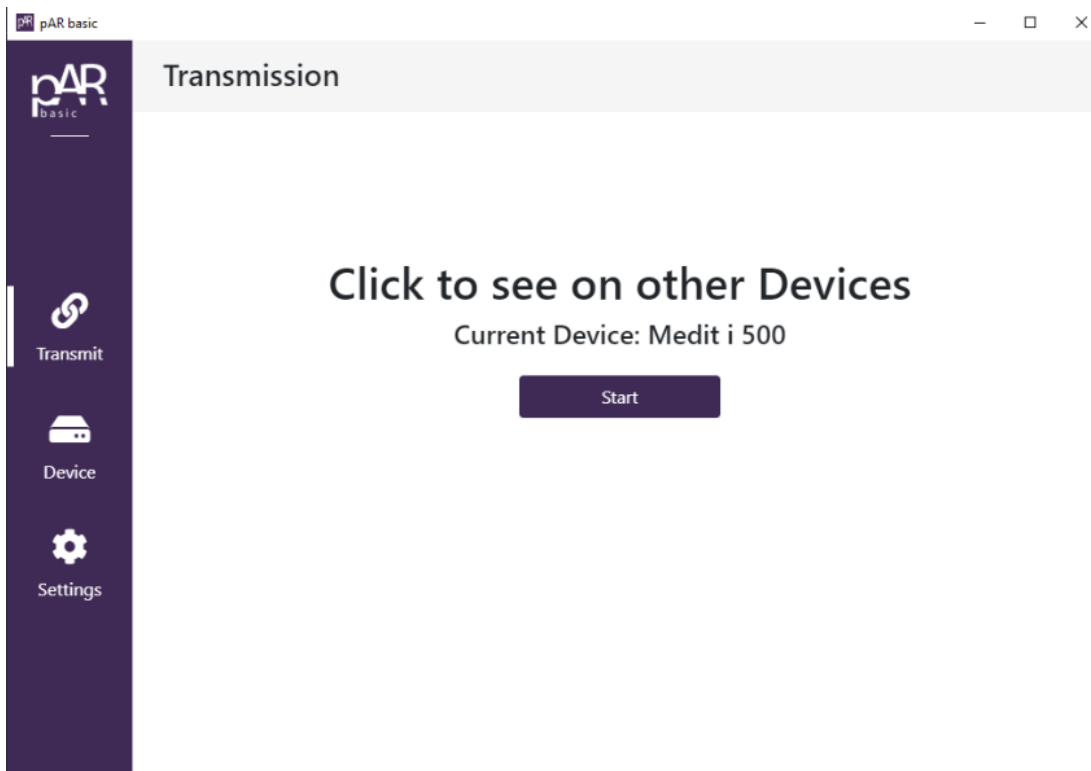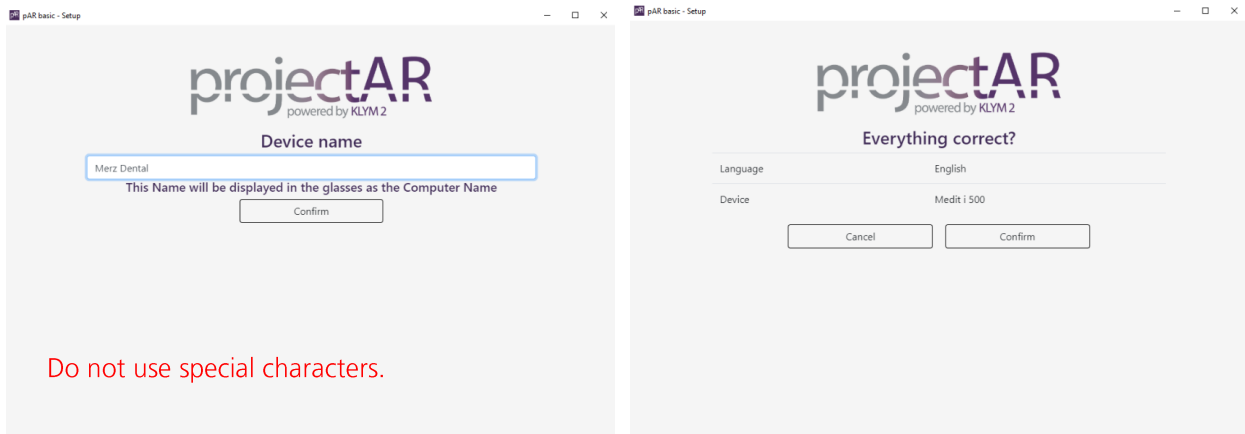


projectAR Installation instructions

Stand 11/2019


Seite 7 von 8

Now connect the glasses and your computer / CAD system to the same WLAN. Set your network to "private" and make sure that there is a good signal strength and stable connection.

Settings computer / CAD-System:

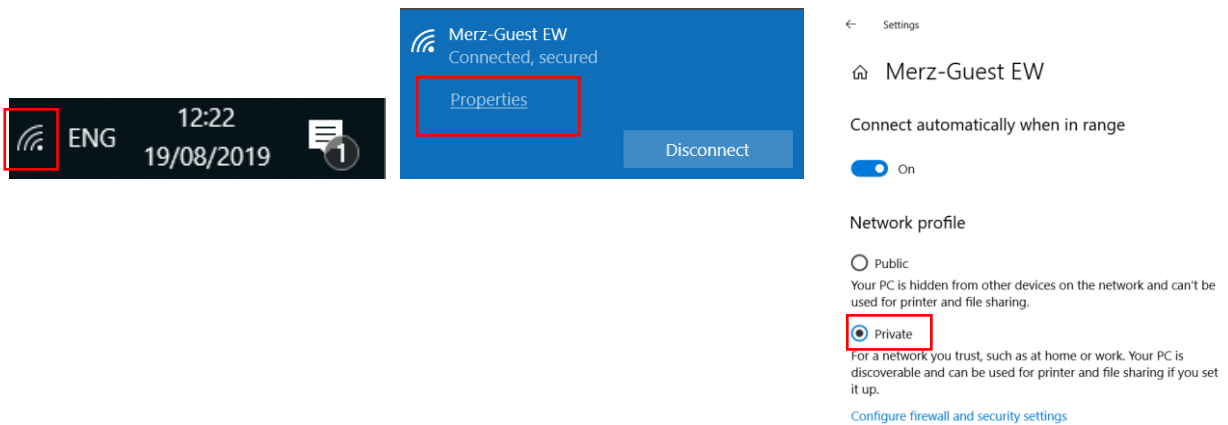

Settings multimedia glasses:


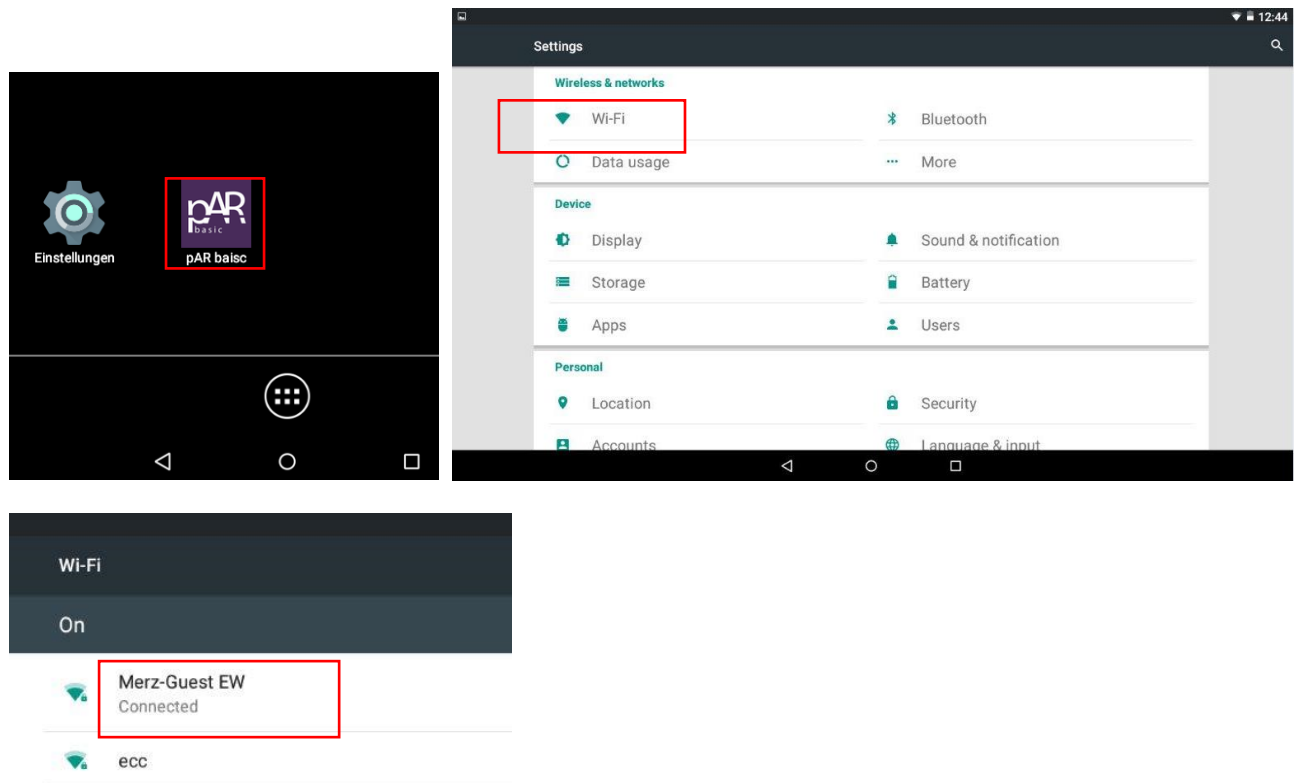


projectAR Installation instructions


Stand 11/2019

Seite 8 von 8

Start the **projectAR** app on the glasses and select your device (computer name) specified when setting up the software.

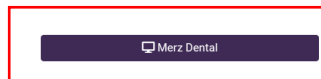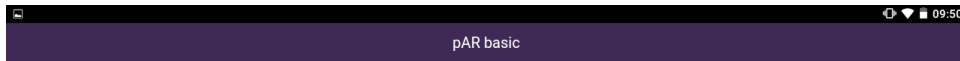

Then, in the software on your computer / CAD system, go to "Start" to start the transfer.

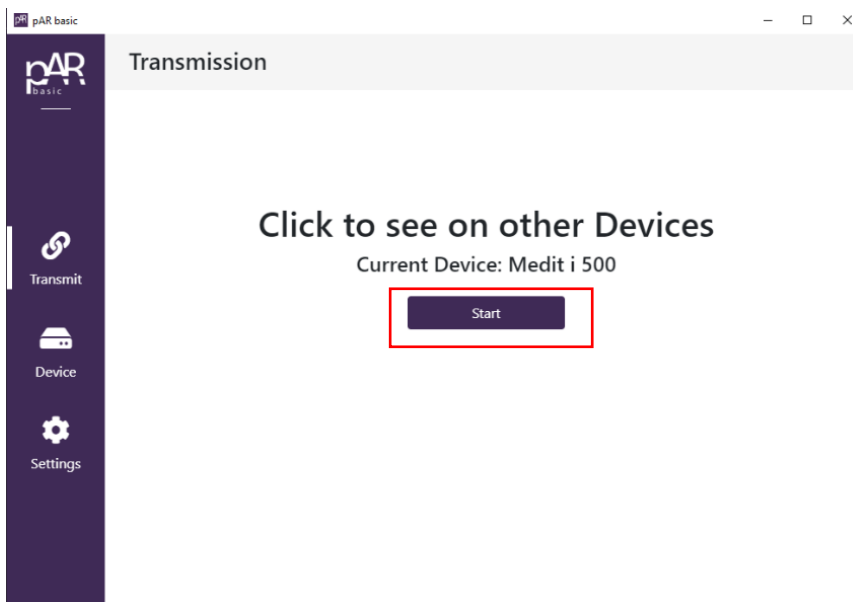

The installation and setup of **projectAR** multimedia glasses is now complete.

Hopefully, you will also find an answer to your question under the **Frequently Asked Questions (FAQ)**. If this is not the case, please do not hesitate to contact us directly.